

DENTAL UNIT AND CHAIR

tbCOMPASS


Parts List



TABLE OF CONTENTS

	page
1 CHAIR BASE COVER SECTION	1
2 CHAIR UPPER SECTION	3
3 CHAIR BASE SECTION	5
4 CHAIR MOTOR SECTION	7
5 CHAIR SWITCH SECTION	9
6 DOCTOR SECTION	11
7 DOCTOR ARM SECTION	13
8 DOCTOR TABLE BRACKET SECTION	15
9 DOCTOR TABLE SECTION	17
10 CUSPIDOR COVER SECTION	19
11 CUSPIDOR FLAME SECTION	21
12 CUSPIDOR P.C.BOARD SECTION	23
13 CUSPIDOR ARM SECTION	25
14 ASSISTANT PANEL SECTION	27
15 CUSPIDOR SHAFT SECTION	29
16 CUSPIDOR VALVE SECTION	31
17 SOLID COLLECTOR SECTION	33
18 ASSISTANT ARM (STANDARD) SECTION	35
19 ASSISTANT ARM (BALANCE ARM TYPE) SECTION	37
20 CUSPIDOR BOWL SECTION	39
21 JUNCTION BOX SECTION	41
22 LIGHT POLE SECTION	43
23 VAC PACK SECTION	45


1. CHAIR BASE COVER SECTION

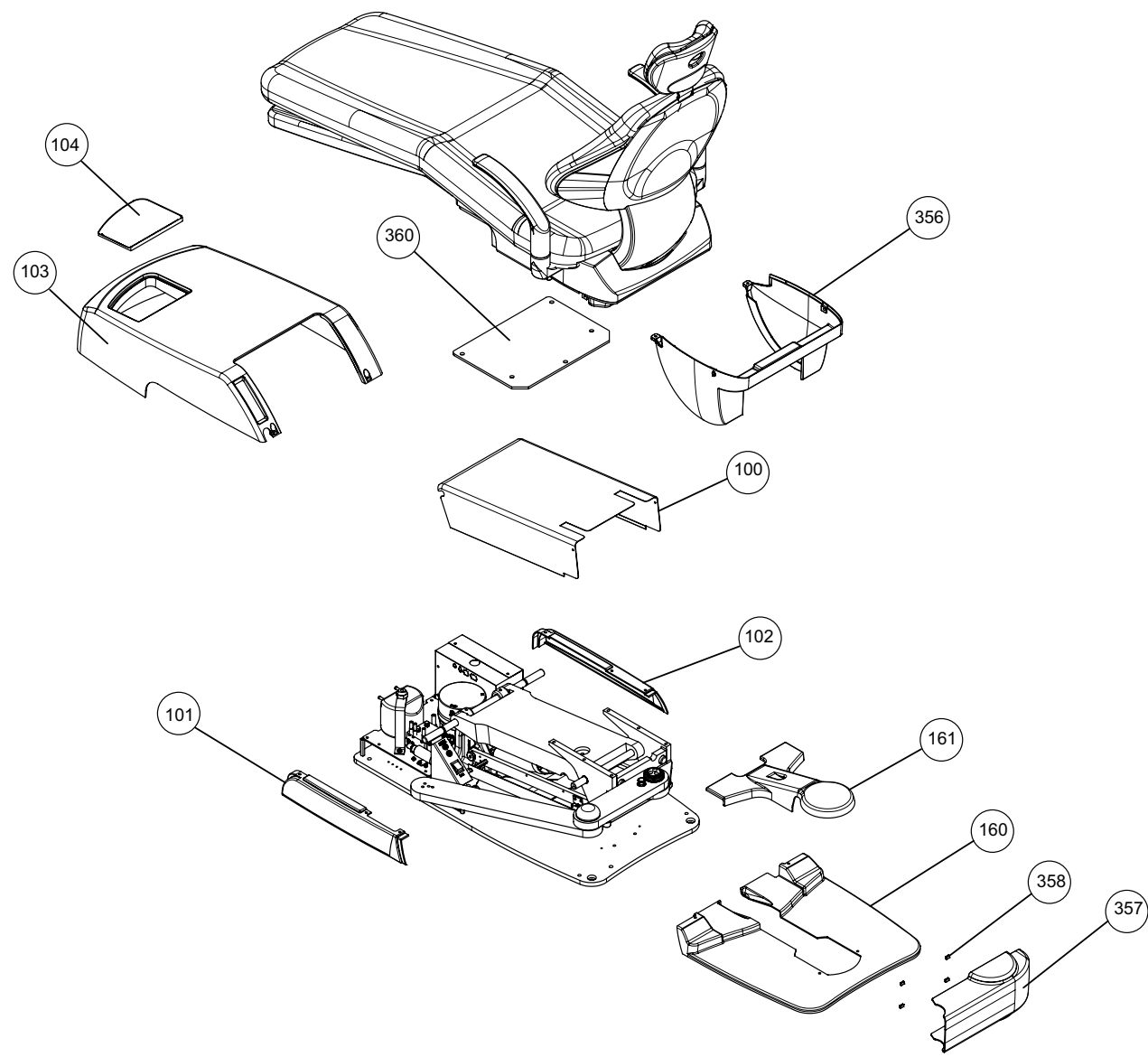

1. CHAIR BASE COVER SECTION

ID No.	Parts No.	Description	QTY
100	F17-DACB01A0	MAIN LINK COVER	1
101	F17-1A09E3A0	BASE COVER LEFT	1
102	F17-1A09E2A0	BASE COVER RIGHT	1
103	F17-DACA02C0	PUMP COVER STANDARD TYPE	1
104	F17-DACJ04B0	MAINTENANCE LID	1
160	F17-1A09F3A0	BASE BACK COVER (GRAY)	1
	F17-1A09FLA0	BASE BACK COVER FOR CUSPIDOR PACKAGE(GRAY)	1
161	F17-1A09E5A0	HOSE COVER	1
356	F17-1A09E4A0	FLANGE COVER (STANDARD TYPE)	1
	F17-DACA03B0	FLANGE COVER (WITHOUT CUSPIDOR TYPE)	1
357	F17-1G036LA0	SWING ARM BRACKET COVER WITH MAGNET	1
358	F17-1A09QAA0	MAGNET	4
360	F17-DACE05B0	MOUNTING SUPPORT PLATE	1


2. CHAIR UPPER SECTION

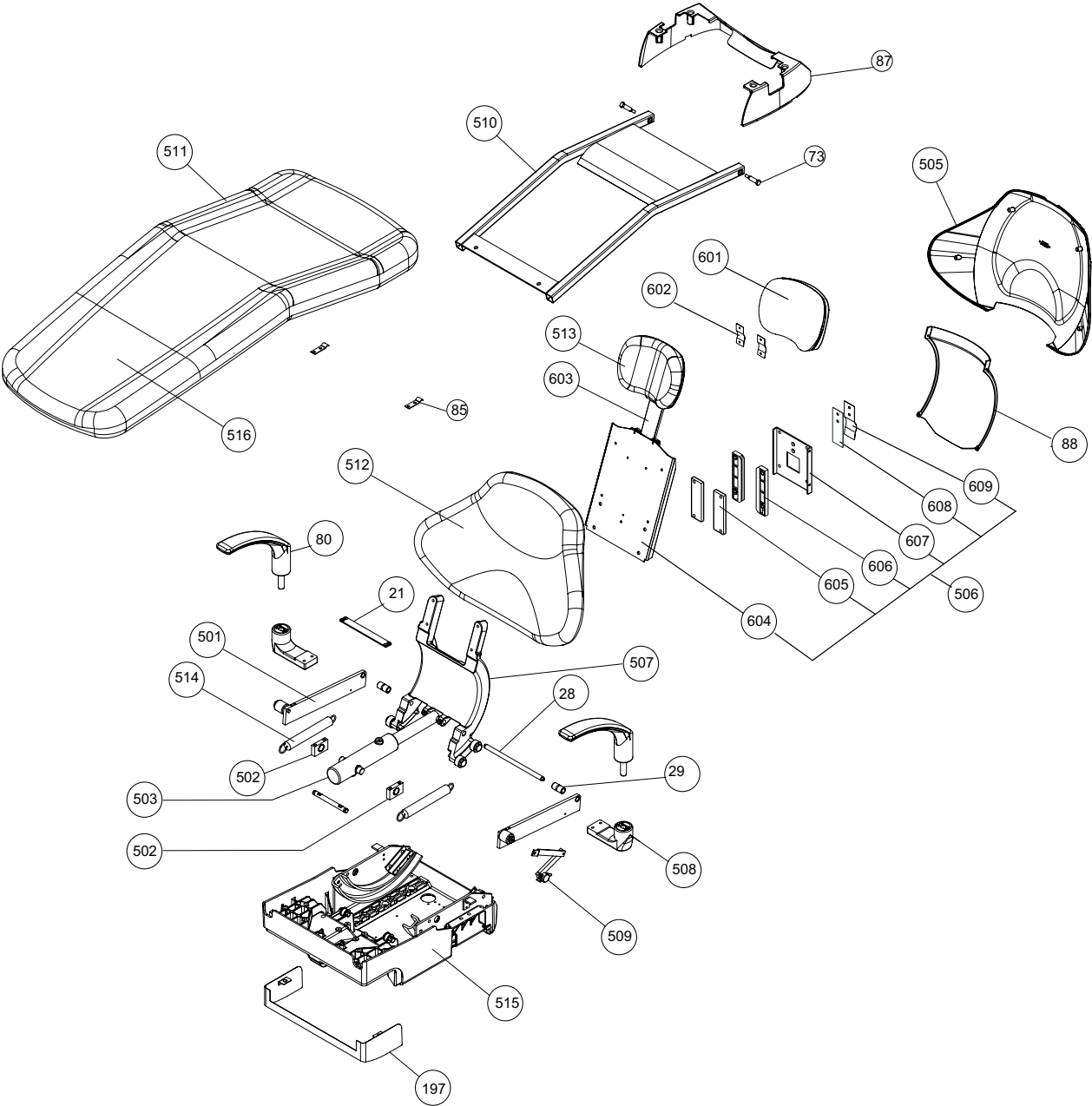

2. CHAIR UPPER SECTION

ID No.	Parts No.	Description	QTY
21	F17-DACR13B0	TILT BAR CONNECTING PLATE	1
28	F17-DATS28A0	RECLINING CYLINDER RAM PIN	1
29	F17-DATT63A0	SPRING COLLAR	2
73	F17-DACR15A0	SHOULDER BOLT M8 FOR SEAT FRAME	2
80	F17-AXEA78A0	ARMREST WITH PIN	2
85	F17-DATS91A0	SEAT FIXING PLATE	2
87	F17-DACA04B0	UPPER FLANGE COVER	1
88	F17-DACB05B0	BACKREST SUPPORT COVER	1
197	F17-1A0AE3A0	LIGHT BRACKET COVER	1
501	F17-AXCA24A0	ROLLER TILT ASSY	2
502	F17-AXBA50A0	RECLINING BRACKET W/BUSH	2
503	F17-AXBA44A0	RECLINING CYLINDER ASSY	1
		WITH AXBA44A0 O-RING P-3 / AANF59B0 RETAINER	1
		RING / ABNL93B0 U-PIN	1
505	F17-AXEA68A0	BACKREST COVER ASSY	1
506	F17-AXCA29A0	BACK SUPPORT PLATE ASSY	1
507	F17-AXCA26A0	BACK SUPPORT ASSY	1
508	F17-DAAE02B0	ARMREST BRACKET	2
509	F17-AXCA28B0	RECLINING POTENTIOMETER ASSY	1
510	F17-AXBA51A0	SEAT FLAME ASSY	1
511	F17-AGFP05AS	SEAT CUSHION W/O UPH	1
	F17-AGFP05A0	SEAT CUSHION	1
512	F17-AGFP04AS	BACKREST CUSHION W/O UPH	1
	F17-AGFP04A0	BACKREST CUSHION	1
513	F17-AGFJ74AS	CLAIR TYPE HEADREST CUSHION W/O UPH	1
	F17-AGFJ74A0	CLAIR TYPE HEADREST CUSHION	1
514	F17-DATS30A0	SPRING	2
515	F17-AXCA25A0	FLANGE ASSY	1
516	F17-AAFB75A0	CLEAR SEAT COVER	1
601	F17-1A0AE40	CLAIR TYPE HEADREST COVER	1
602	F17-AAER61A0	COVER CLIP	2
603	F17-AGEK02A0	TWIN AXIS HEADREST BAR ASSY	1
604	F17-DACE10B0	BACKREST PLATE	1
605	F17-DAQR14B0	HEADREST BAR GUIDE SPACER	2
606	F17-U-35340	HEADREST BAR GUIDE	2
607	F17-DAQJ66A0	HEADREST BAR GUIDE BRACKET	1
608	F17-DACR19A0	HEADREST BAR RUBBING PLATE	1
609	F17-DACR17A0	HEADREST BAR ADJUSTING SPRING	1


3. CHAIR BASE SECTION

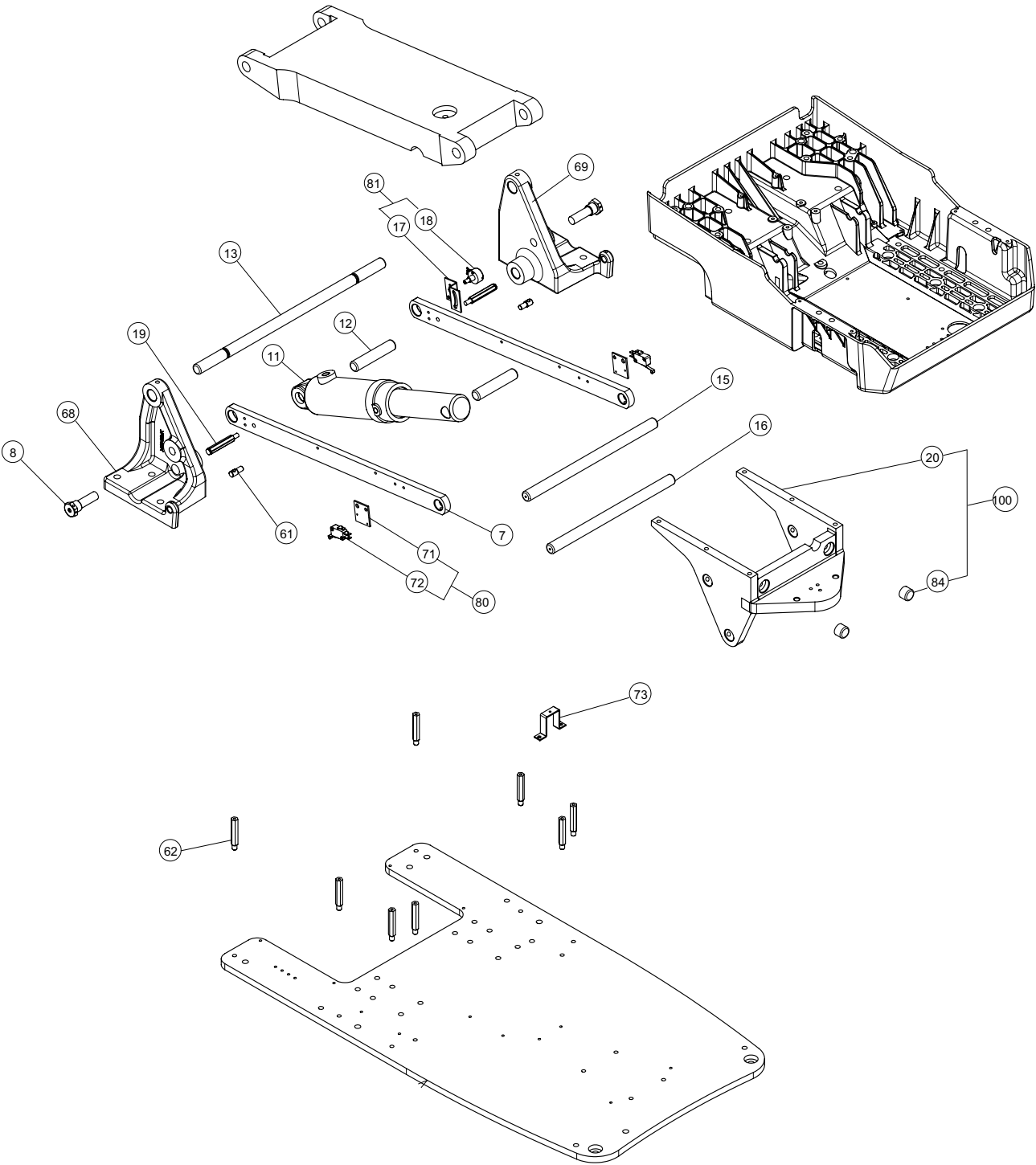

3. CHAIR BASE SECTION

ID No.	Parts No.	Description	QTY
7	F17-AXBA48A0	SUB LINK W/(9)BUSH	2
8	F17-G-01230	SUB LINK SHOULDER BOLT	2
11	F17-1G031AA0	BASE CYLINDER ASSY WITH BUSH RUM	1
12	F17-R-31950	BASE CYLINDER RAM PIN	2
13	F17-DACJ07B0	LINK BRACKET SHAFT	1
15	F17-DACJ09A0	MAIN LINK SHAFT	1
16	F17-DACJ10A0	SUB LINK SHAFT	1
17	F17-AAET12A0	BRACKET FOR POTENTIOMETER	1
18	F17-1A04KNA0	POTENTIOMETER ASSY W/WIRE	1
19	F17-DACR10B0	SPACER 68L	2
20	F17-1G0316A0	MOUNTING BRACKET FLANGE ASSY (W/(75)BUSH)	1
61	F17-DACR09B0	SUB LINK COVER PIN	2
62	F17-1A09E1A0	BASE COVER SPACER	8
68	F17-1G0318A0	LINK BRACKET LEFT W/(6)BUSH	1
69	F17-1G0319A0	LINK BRACKET RIGHT W/(6)BUSH	1
71	F17-G-00870	BRACKET FOR LOCK SWITCH	1
72	F17-DNBR78A0	MICRO SWITCH V-5515EK	2
73	F17-DATS73B0	HOSE COVER BRACKET	1
80	F17-1G01SFA0	CHAIR LOCK SWITCH ASSY W/(71)BRACKET	2
81	F17-AXCA27B0	POTENTIOMETER W/(17)BRACKET	1
84	F17-DAMS68AA	RUBBER STOPPER	2
100	F17-1G0316A0	FLANGE ASSY W/RUBBER STOPPER	1


4. CHAIR MOTOR SECTION

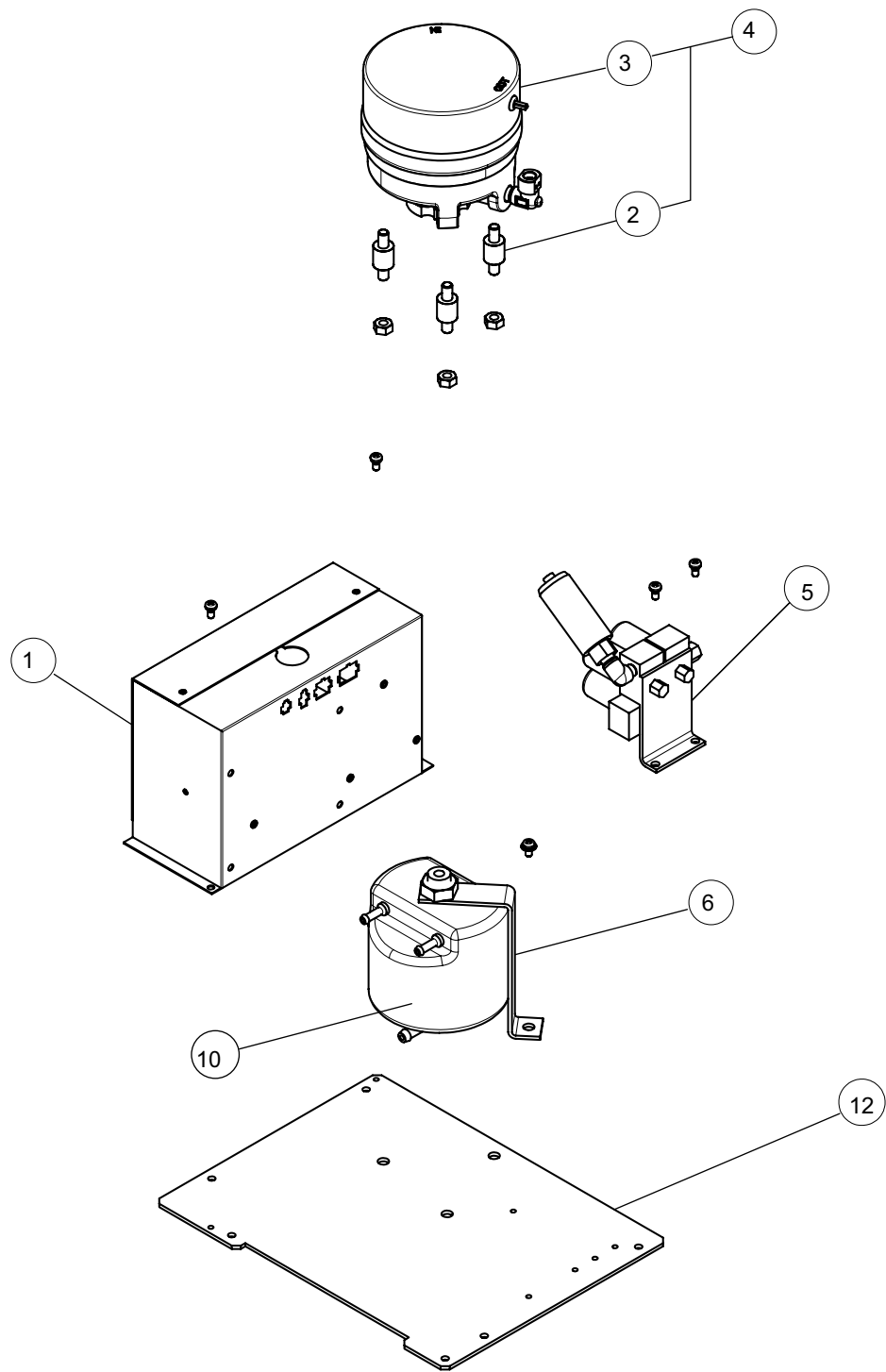

4. CHAIR MOTOR SECTION

ID No.	Parts No.	Description	QTY
1	F17-1G01XVB0	CONTROL BOX230V ASSY CONTROL BOX ASSY	1
	F17-AXAA90C0	CONTROL BOX 115V ASSY CONTROL BOX ASSY	1
	F17-AXAA94C0	CONTROL BOX 220V ASSY CONTROL BOX ASSY	1
	F17-AXAA99C0	CONTROL BOX 240V ASSY CONTROL BOX ASSY	1
	F17-1G01GZF0	PLATFORM T5 CONTROL PCB (220V-240V)	1
	F17-1G01H0E0	PLATFORM T5 CONTROL PCB (100V-120V)	1
2	F17-AAND52B0	CUSHION RUBBER	4
3	F17-ANER60A0	MOTOR PUMP 220-240V	1
	F17-ANET56A0	MOTOR PUMP 115V	1
4	F17-1G0315B0	MOTOR PUMP ASSY 220-240V(W/CUSHION RUBBER)	1
	F17-1G03NYA0	MOTOR PUMP ASSY 115V(W/CUSHION RUBBER)	1
5	F17-AXBB46A0	SOLENOID VALVE BLOCK 230-240V	1
	F17-AXBB45A0	SOLENOID VALVE BLOCK 220V	1
	F17-AXBB43A0	SOLENOID VALVE BLOCK 115V	1
6	F17-AXBA42A0	OIL RESERVOIR W/BACKET ASSY	1
10	F17-AAFE57A0	OIL RESERVOIR	1
12	F17-1A09DZA0	MOUNTING PLATE	1


5. CHAIR SWITCH SECTION

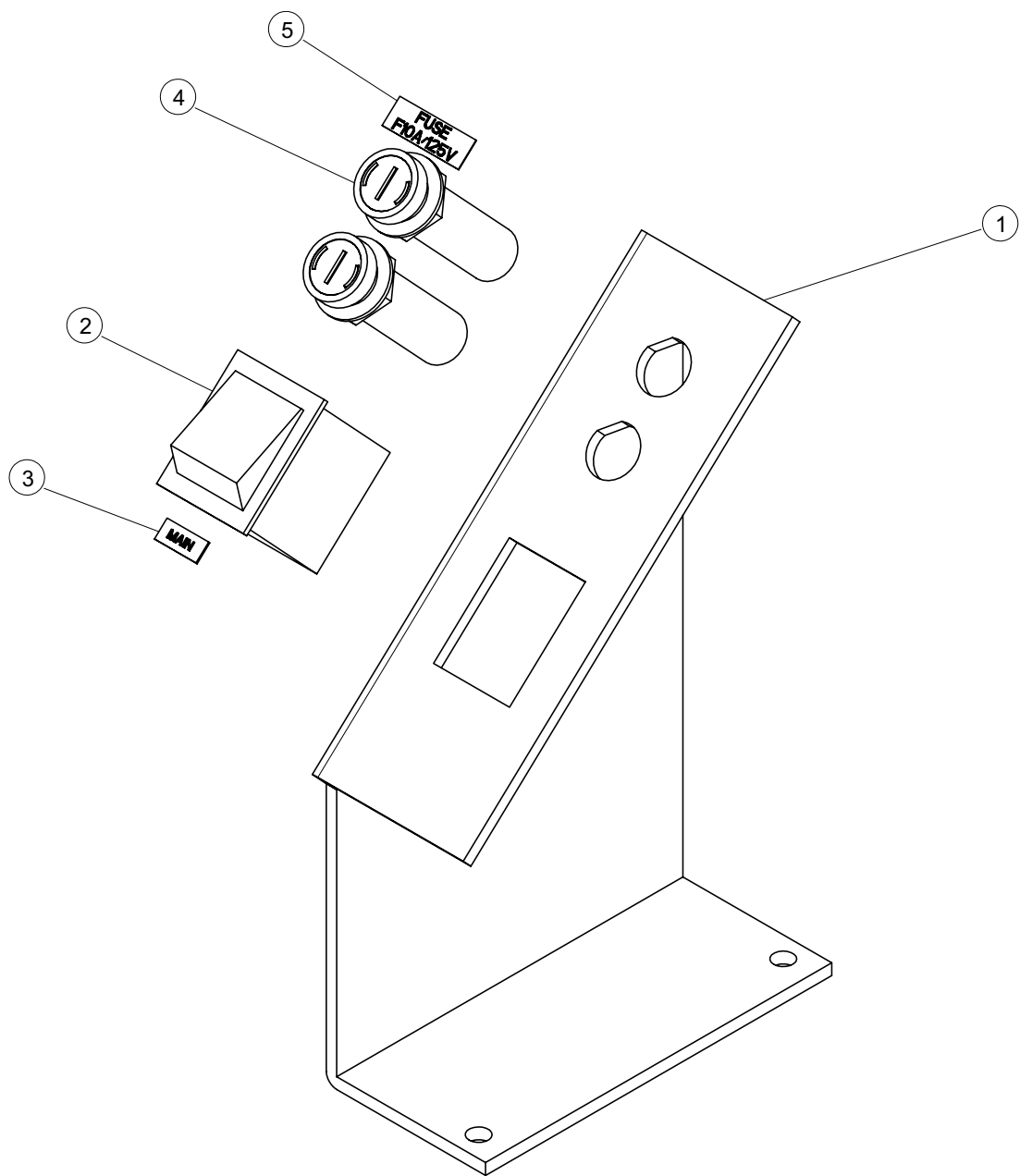

5. CHAIR SWITCH SECTION

ID No.	Parts No.	Description	QTY
1	F17-1A09E0A0	MAIN SWITCH PANEL	1
2	F17-DETR24A0	MAIN SWITCH A8A-223-1 (220-240V)	1
	F17-BELA20A0	MAIN SWITCH A8A-213-1(110-120V)	1
3	F17-X-61470	MAIN SWITCH MARK (MAINS)	1
4	F17-AEFR93A0	FUSE HOLDER MF-530(110-120V)	2
	F17-AEFR92A0	FUSE HOLDER F-400-01A2 (220-240V)	2
	F17-DERR49A0	FUSE 5A 220-240V	2
	F17-AEFW14A0	FUSE 10A 100-120V	2
5	F17-DARS01A0	FUSE LABEL F5A/250V	1
	F17-G-00520	FUSE LABEL F10A/125V	1


6. DOCTOR SECTION

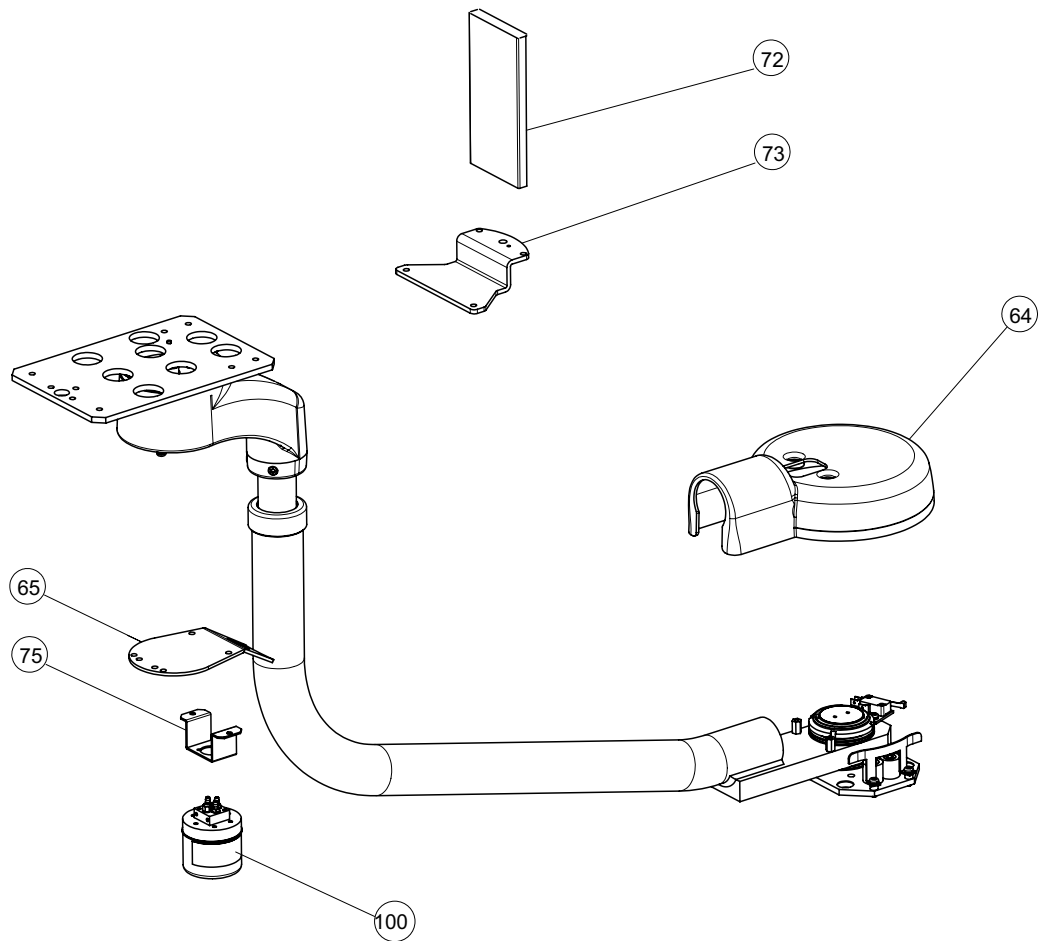

6. DOCTOR SECTION

ID No.	Parts No.	Description	QTY
64	F17-1A09EZA0	DOCTOR TABLE ARM COVER	1
65	F17-1A09F0A0	DOCTOR TABLE BRACKET COVER	1
72	F17-FACC39A0	VIEWER BRACKET	1
73	F17-1A09FDA0	VIEWER PLATE	1
75	F17-1A09FEA0	OIL MIST SEPARATOR BRACKET	1
100	F17-FGRA04B0	OIL MIST SEPARATOR ASSY	1


7. DOCTOR ARM SECTION

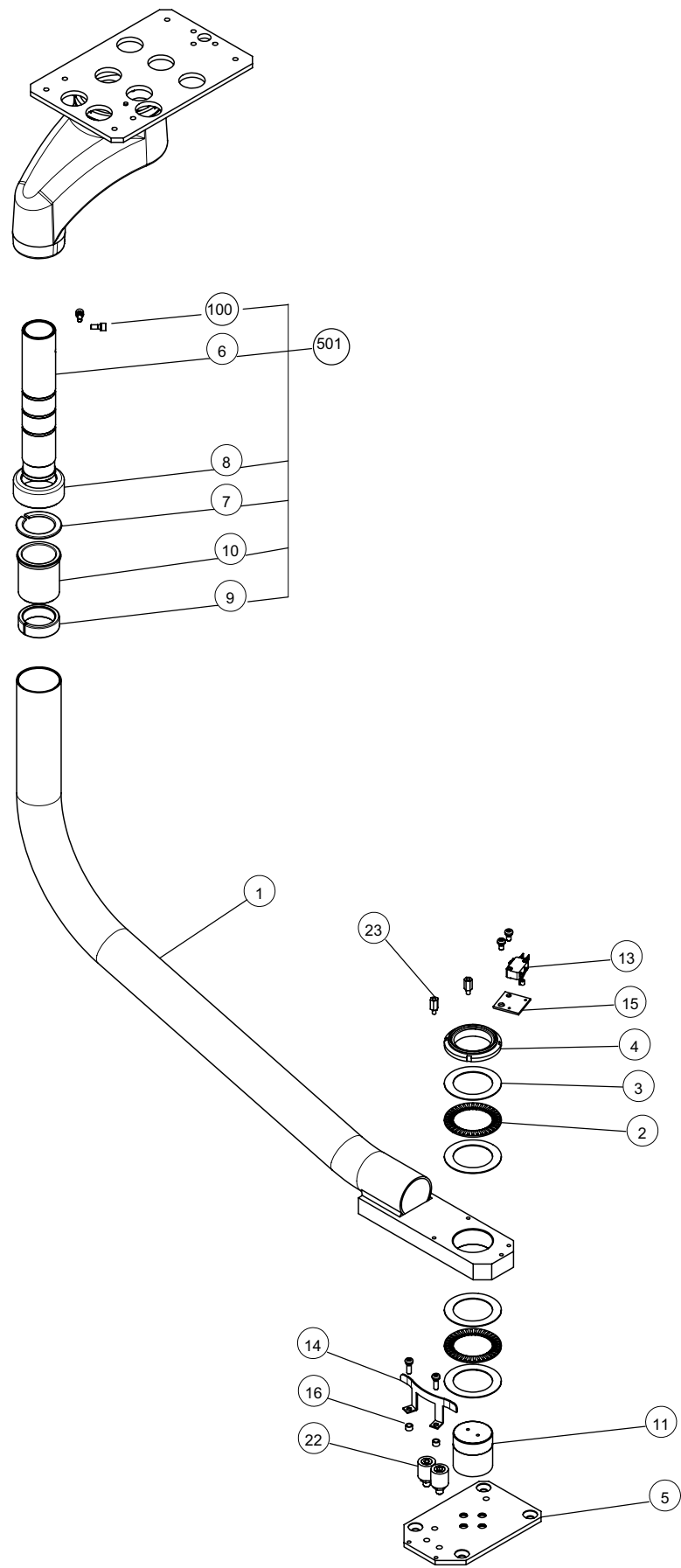

7. DOCTOR ARM SECTION

ID No.	Parts No.	Description	QTY
1	F-171A09DJB0	DR.TABLE LOWER SWING ARM	1
2	F-17FJNA55A0	NEEDLE BEARING AXK-1109	2
3	F-17FJNA56A0	THRUST WASHER AS1109	4
4	F-17FJTA26A0	FINE U-NUT FU09SS M45	1
5	F-171A09DPA0	DR.TABLE LOWER SWING ARM BRACKET	1
6	F-171A09DKB0	UPPER ARM	1
7	F-17DAQR33A0	LOCK RING	1
8	F-17DAQR34B0	GUIDE COLLAR	1
9	F-17DAQR35A0	GUIDE SLEEVE	1
10	F-17DAQR36A0	ARM SLEEVE	1
11	F-171A0658A0	DR.TABLE LOWER SWING ARM SHAFT	1
13	F-17DNBR78A0	MICRO SWITCH V-5515EK	1
14	F-17DATS72B0	SWITCH PLATE	1
15	F-171A09DQA0	SWITCH BRACKET	1
16	F-17AFDR19B0	SPACER 5-6L	2
22	F-17U-06690	DR.TABLE LOWER SWING ARM RUBBER STOPPER	2
23	F-17DATS80B0	DR.TABLE SIDE LOWER SWING ARM SPACER	2
100	F-17ZA765NRU	CAP BOLT M5-10	2
501	F-171P033PA0	UPPER ARM ASSY	1


8. DOCTOR TABLE BRACKET SECTION

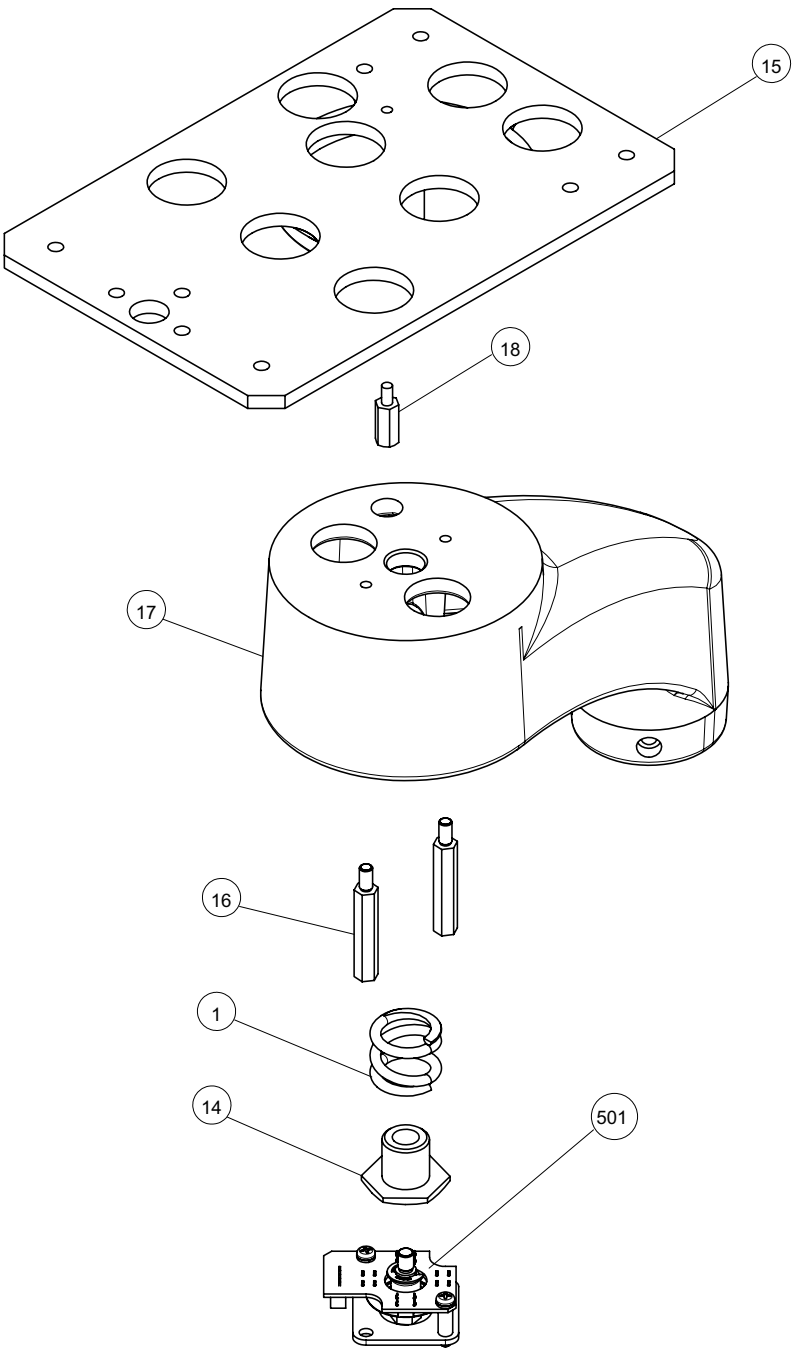

8. DOCTOR TABLE BRACKET SECTION

ID No.	Parts No.	Description	QTY
1	F17-1A09DLA0	TABLE SUPPORT SPRING	1
14	F17-1A09DRA0	SPRING GUIDE	1
15	F17-1A09DTA0	TABLE BOTTOM PLATE	1
16	F17-1E02V9A0	SPACER BSB-540E	2
17	F17-1A09DWA0	TABLE BRACKET	1
18	F17-FACA40A0	ROTATION STOPPER BOLT	1
501	F17-1G031KA0	SAFETY SWITCH ASSY	1


9. DOCTOR TABLE SECTION

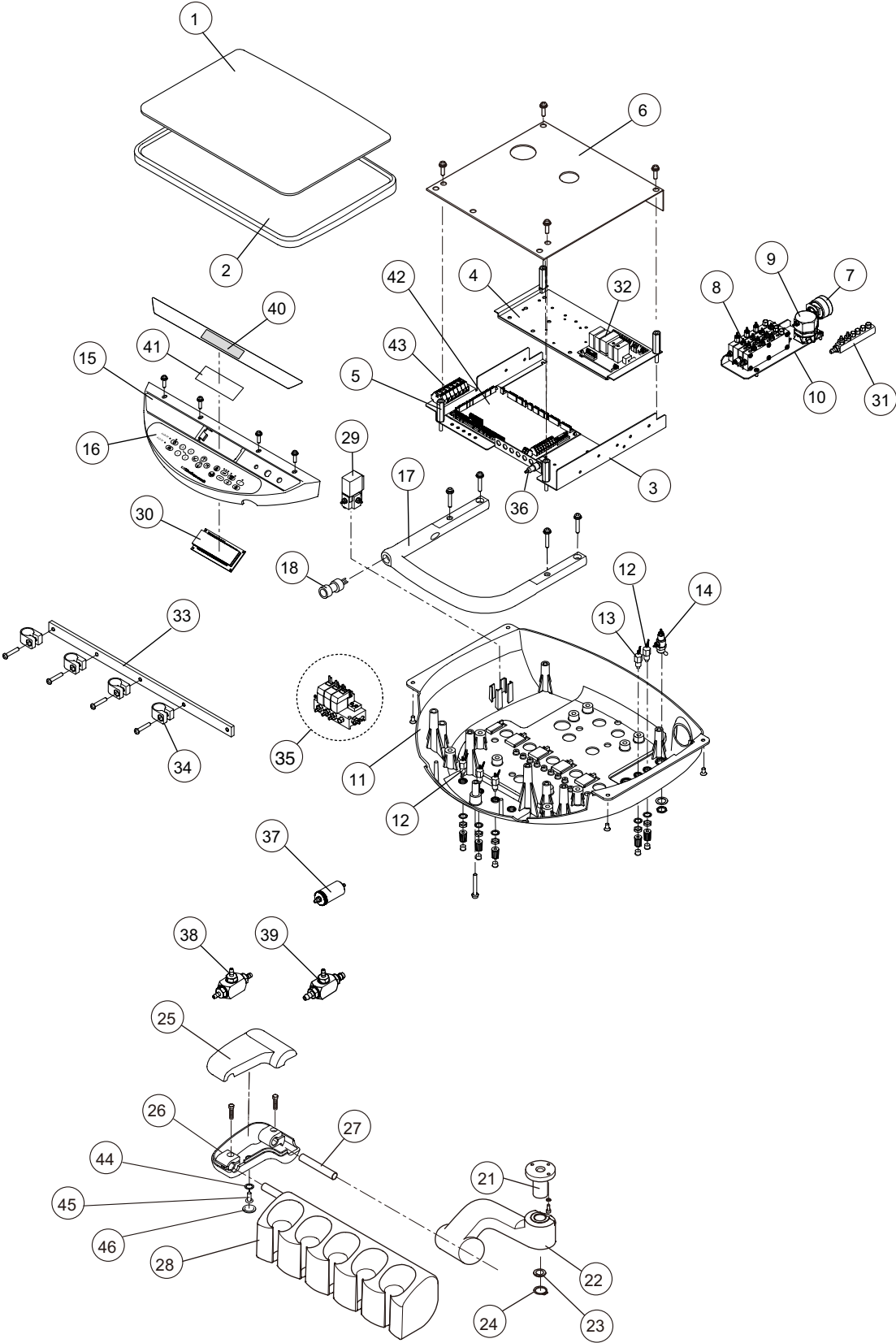

9. DOCTOR TABLE SECTION

ID No.	Parts No.	Description	QTY
1	F17-FACB93A0	TABLE TRAY MAT (S)	1
2	F17-FACA85B0	DOCTOR TABLE TRAY (S)	1
3	F17-FACB03E0	P.C.BOARD BRACKET	1
4	F17-FADA04C0	MICRO MOTOR P.C.BOARD BRACKET	1
5	F17-1E00CXA0	SPACER (BSB-436E)	3
6	F17-AADF06A0	P.C.BOARD BRACKET COVER	1
7	F17-DESR15B0	HANDPIECE PRESSURE GAUGE (18-030-005-ZA)	1
8	F17-1G009DC0	AUTO SELECT VALVE BLOCK ASSY	1
9	F17-AAEK28C0	PULL BACK VALVE ASSY	1
10	F17-FACB02A0	AUTO SELECT VALVE BRACKET	1
11	F17-FACA82B0	TABLE HOUSING	1
12	F17-FAOA09A0	NEEDLE VALVE ASSY FOR WATER	1
13	F17-FGOA10A0	NEEDLE VALVE ASSY FOR AIR	1
14	F17-DELR25AA	3V VALVE ASSY	1
15	F17-FACA83D0	MEMBRANE SWITCH PANEL	1
16	F17-FADB45B0	MEMBRANE SWITCH	1
17	F17-FACA88A0	TABLE HANDLE	1
18	F17-ANDT11A0	AIR BRAKE SWITCH	1
19	F17-AAEJ27A0	LEVEL ADJUSTING PLATE	1
20	F17-AADB91A0	TABLE BRACKET	1
21	F17-FAQA95C1	H/P HOLDER ARM ROTATION SHAFT	1
22	F17-1A010RA0	HOLDER BRACKET	1
23	F17-FAQA97A0	H/P HOLDER BRACKET WASHER	1
24	F17-FJRA03A0	C-RING (C18)	1
25	F17-1A03PUA0	H/P HOLDER ARM COVER	1
26	F17-1A03PTA0	H/P HOLDER ARM	1
27	F17-FACB91B0	H/P HOLDER BAR	1
28	F17-FGDA02B0	5 H/P HOLDER ASSY	1
29	F17-ASLA06A0	BALANCE ARM BRAKE SOLENOID VALVE	1
30	F17-FEDA03D0	DISPLAY ASSY	1
31	F17-ASLA24A0	MANIFOLD	1
32	F17-FEDA02B0	INTERFACE RELAY P.C.BOARD	2
33	F17-FACB06A0	COVER HOUSE CLAMP BRACKET	1
34	F17-FALD62B0	COVER HOSE CLAMP	4
35	F17-ASLA07A0	SOLENOID VALVE BLOCK FOR PILOT	1
36	F17-ASMA13A0	MINI AIR SWITCH ASSY	1
37	F17-FGMA18A0	IN LINE FILTER ASSY	1
38	F17-FGMA97A0	T-JOINT (BF1/8-3)	1
39	F17-FALC95A0	T-JOINT	1
	F17-AAES69A0	BARB FITTING (1/4)	2
	F17-AAES68A0	BARB FITTING (1/8)	1
	F17-AAES67A0	NYLON WASHER (M5)	3
40	F17-FACB07D0	DISPLAY FILTER	1
41	F17-FACB10B0	DISPLAY COVER	1
42	F17-1G019GA0	DOCTOR TABLE P.C.BOARD	1
43	F17-AGDR39A0	MAIN GAUGE CLAMP	1
44	F17-DEAS14A0	SET CAP WASHER	1
45	F17-ZA112LSU	SCREW M4-12 SUS	1
46	F17-DEAS15A0	SET CAP COVER (WHITE)	1


10. CUSPIDOR COVER SECTION

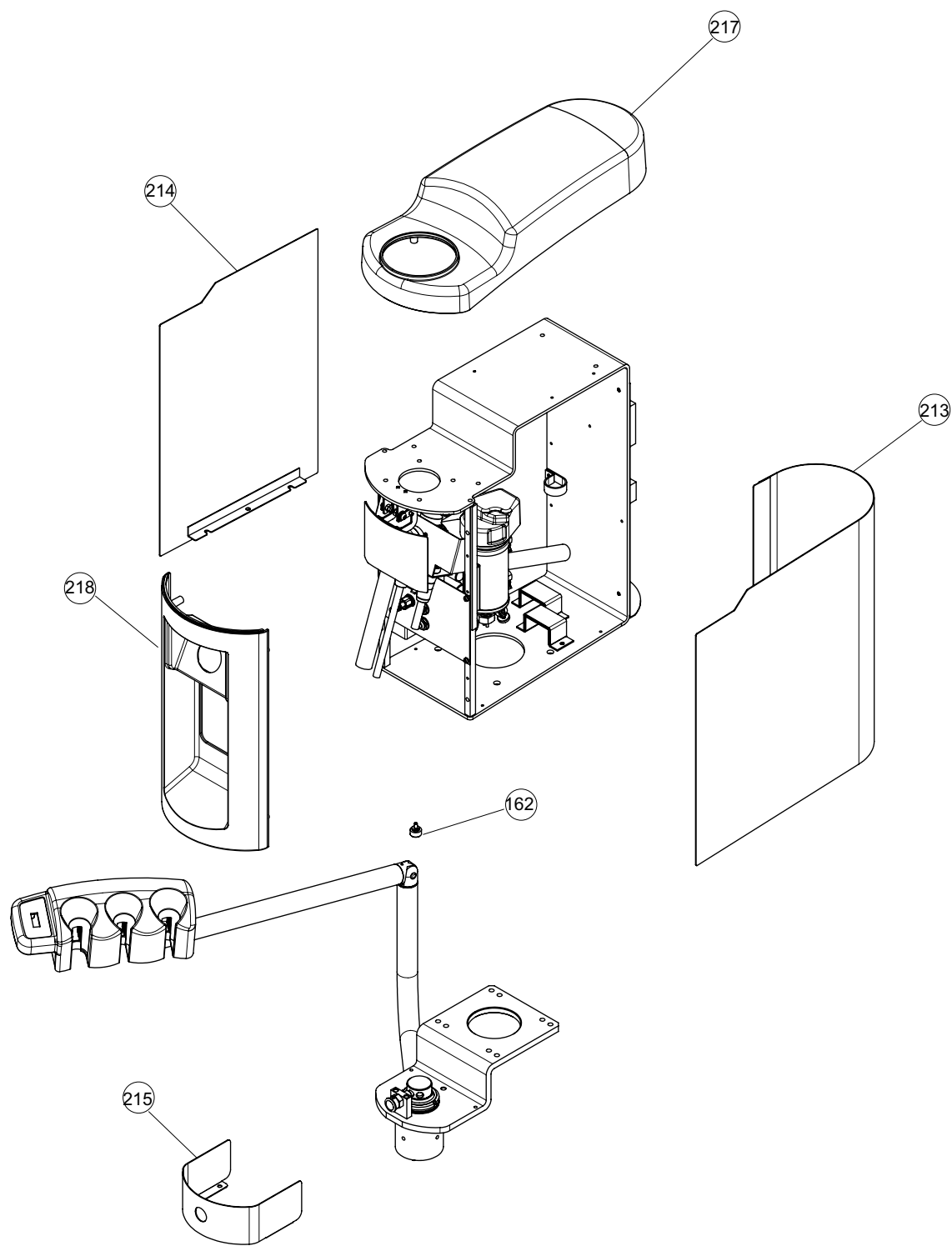

10. CUSPIDOR COVER SECTION

ID No.	Parts No.	Description	QTY
162	F17-1E01RPA0	KNOB FASTENER	1
213	F17-1A09F6A0	MAINTENANCE PANEL RIGHT	1
214	F17-1A09F5A0	MAINTENANCE PANEL LEFT	1
215	F17-1A09EYA0	CUSPIDOR SHAFT COVER	1
217	F17-1A09FNA0	CUSPIDOR TOP COVER	1
218	F17-1G036GA0	CUSPIDOR FRONT COVER ASSY (W/SPACER BSB-420E)	1


11. CUSPIDOR FLAME SECTION

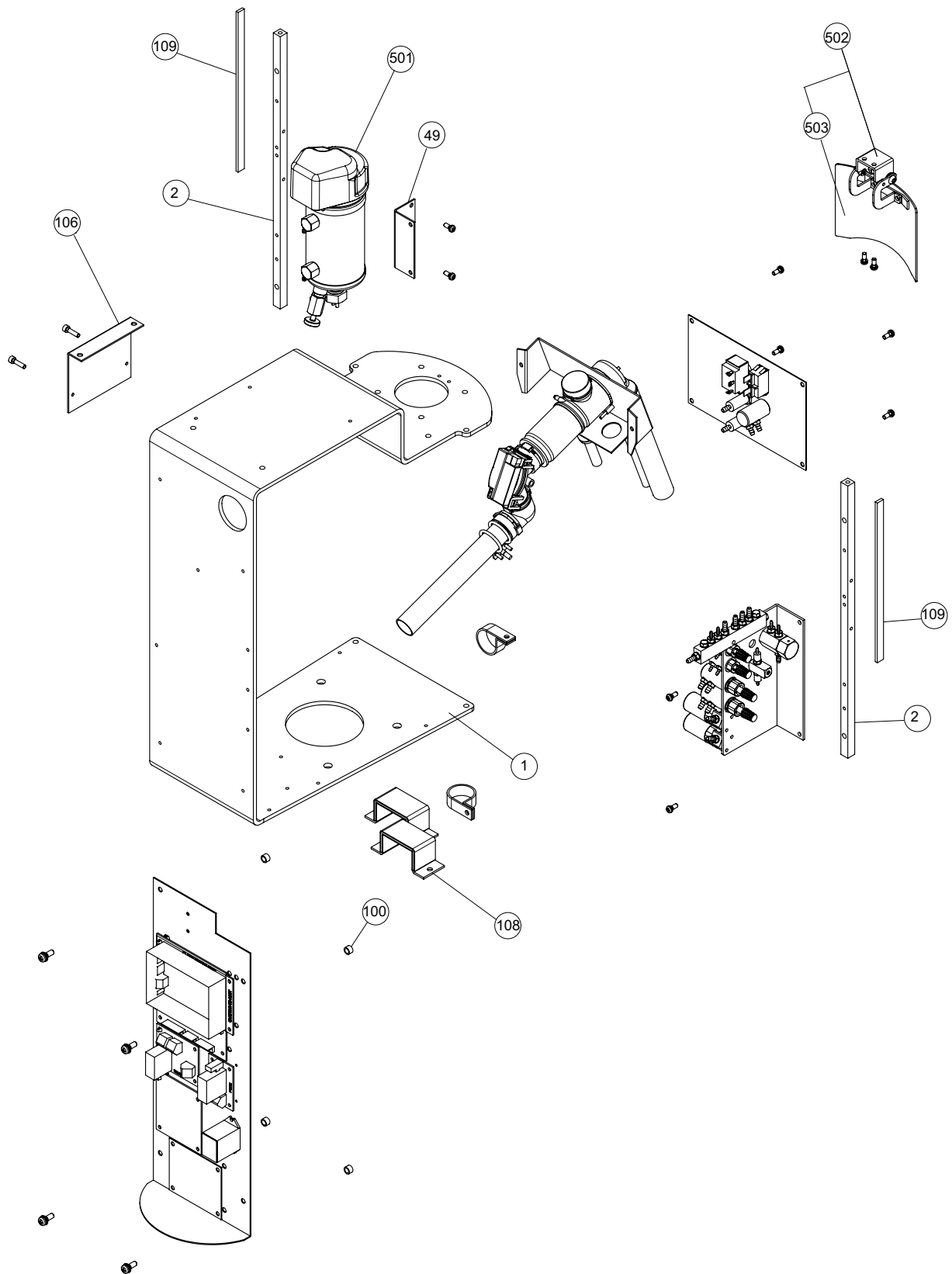

11. CUSPIDOR FLAME SECTION

ID No.	Parts No.	Description	QTY
1	F17-1A09EGA0	CUSPIDOR FLAME	1
2	F17-1A09ELA0	SUPPORTING POST	2
49	F17-AADS63A0	WATER HEATER BRACKET	1
100	F17-X-59140	SPACER 4-5L	4
106	F17-1A09EQA0	BRACKET FOR CS1 / CAS1	1
108	F17-AAET10A0	DRAIN HOSE BRACKET	2
109	F17-AADS43B0	FELT A	2
501	F17-FAMB11F0	WATER HEATER	1
502	F17-1G0375A0	SOLID COLLECTOR COVER ASSY	1
503	F17-1A09EVA0	COVER ONLY SOLID COLLECTOR COVER	1


12. CUSPIDOR P.C.BOARD SECTION

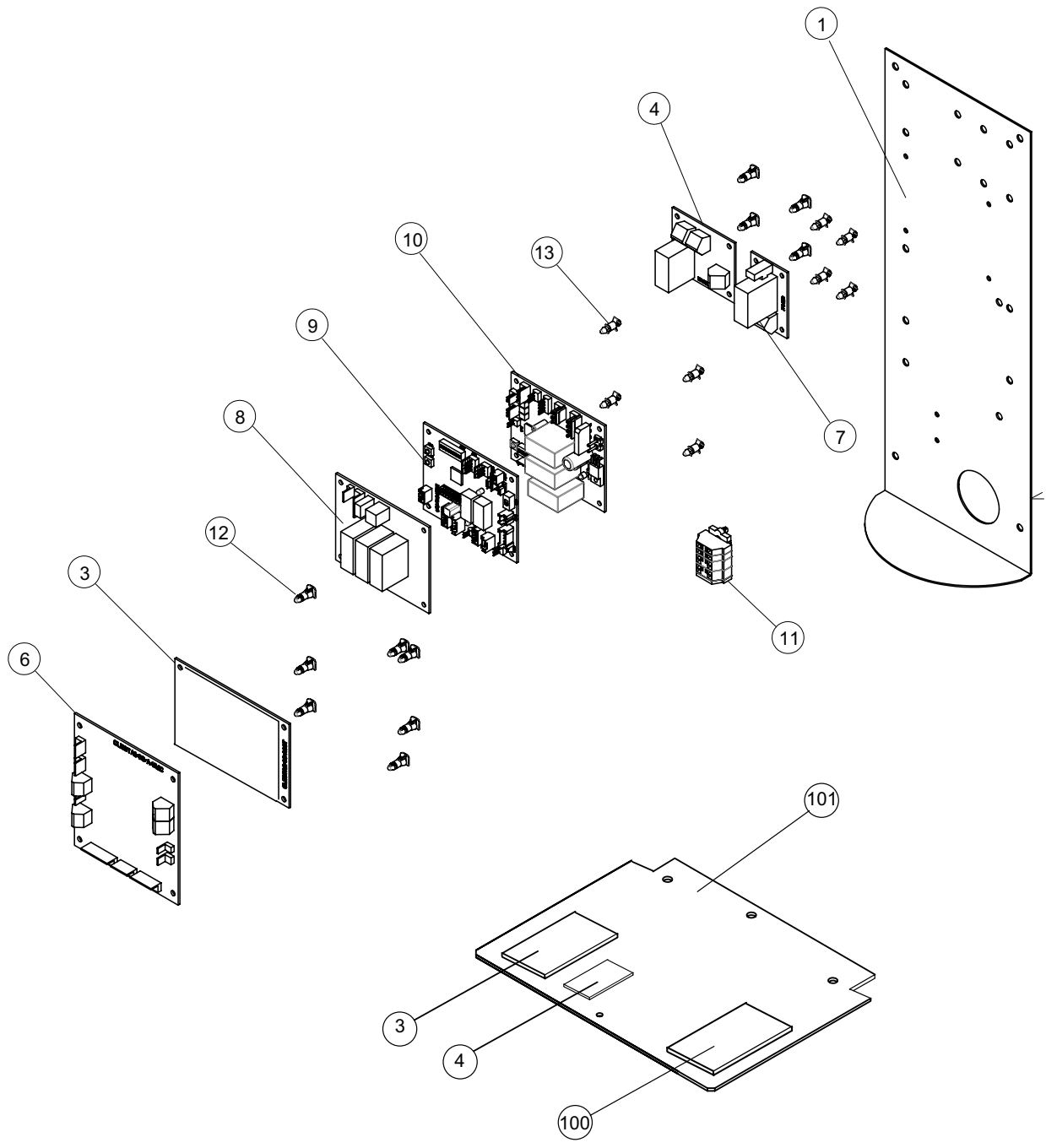

12. CUSPIDOR P.C.BOARD SECTION

ID No.	Parts No.	Description	QTY
1	F17-1A09EPA0	BRACKET FOR P.C.BOARD	1
3	F17-AGDJ73A0	ASSISTANT P.C.BOARD	1
4	F17-DAMK01AA	WATER HEATER RELAY P.C.BOARD	1
6	F17-FECA34E0	CUSPIDOR P.C.BOARD	1
7	F17-X-64180	EXTRA RELAY P.C.BOARD	2
8	F17-CGCN36A0	SENSOR CONTROL P.C.BOARD FOR 720 DENTAL LIGHT	1
9	F17-1G02A0D0	SENSOR CONTROL P.C.BOARD FOR 900 DENTAL LIGHT	1
10	F17-CGGN65A0	SENSOR CONTROL P.C.BOARD FOR 820 DENTAL LIGHT	1
11	F17-1G00QUA0	TERMINAL	1
12	F17-DNOR11A0	SPACER RSPLS-6LU	8
13	F17-CERN39A0	SPACER WLS-04-0	4
100	F17-CGCN36A0	SENSOR CONTROL P.C.BOARD FOR 720 DENTAL LIGHT	1
	F17-1G02A0D0	SENSOR CONTROL P.C.BOARD FOR 900 DENTAL LIGHT	1
	F17-CGGN65A0	SENSOR CONTROL P.C.BOARD FOR 820 DENTAL LIGHT	1
101	F17-1A0AE2A0	MODULE PLATE FOR DOCTOR PACKAGE / VACUUM PACKAGE	1


13. CUSPIDOR ARM SECTION

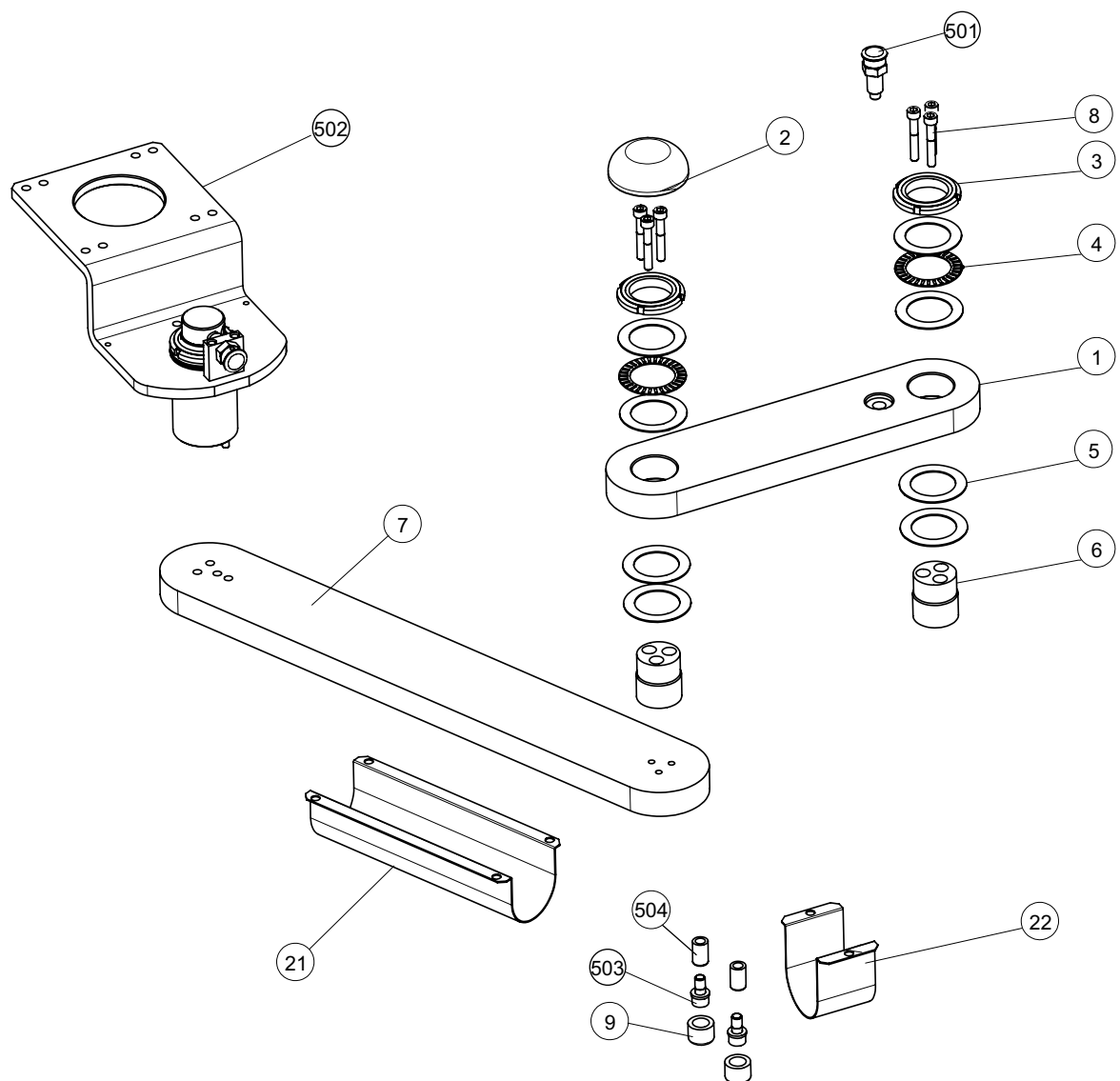

13. CUSPIDOR ARM SECTION

ID No.	Parts No.	Description	QTY
1	F17-1A09E9A0	CUSPIDOR FIRST ARM	1
2	F17-AAFT58A0	RUBBER CAP	1
3	F17-X-59080	FAIN NUT FU07SS M35	2
4	F17-X-59090	THRUST BEARING	2
5	F17-X-59100	THRUST WASHER	4
6	F17-AAFT65A0	ROTATION SHAFT	2
7	F17-1A09EEA0	CUSPIDOR SECOND ARM	1
8	F17-ZA761O3G	SCREW M6-45	6
9	F17-DAMS68AA	STOPPER RUBBER	2
21	F17-1A09Q8A0	HOSE COVER NO.2	1
22	F17-1A09Q7A0	HOSE COVER NO.1	1
501	F17-1G032DA0	LOCK PIN	1
502	F17-1G03S2A0	CUSPIDOR SHAFT ASSY	1
503	F17-ZA761P#Y	SCREW M8-15	2
504	F17-AAET81A0	STOPPER COLLAR	2


14. ASSISTANT PANEL SECTION

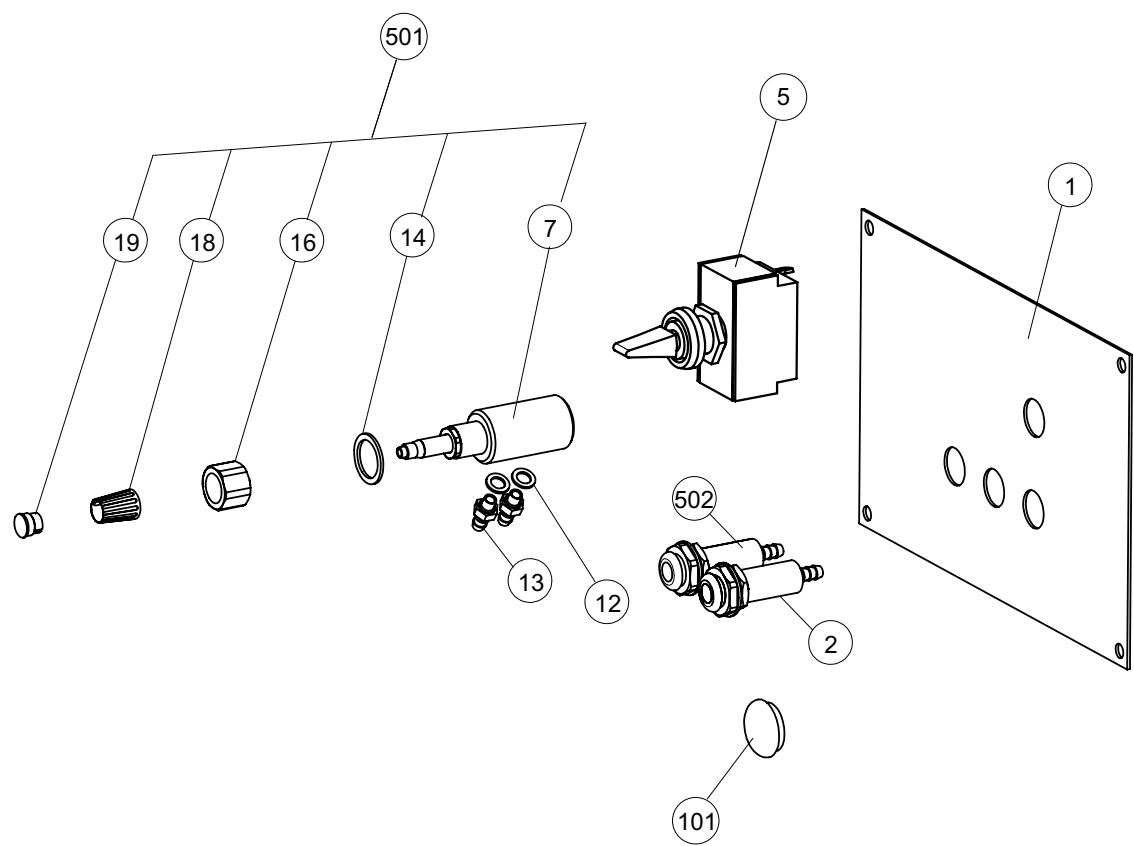

14. ASSISTANT PANEL SECTION

ID No.	Parts No.	Description	QTY
3	F17-X-59080	NUT	1
4	F17-X-59090	BEARING	1
5	F17-X-59100	WASHER	4
10	F17-1A09EAB0	CUSPIDOR SHAFT	1
11	F17-1A09EBA0	BRACKET FOR ARM	1
19	F17-1A09FFA0	LOCK SHAFT BRACKET	1
20	F17-ZG081NWG	ROLL PIN M5-20	1
501	F17-1G031CA0	CUSPIDOR ARM ASSY	1
502	F17-1G031DA0	ARM LOCK ASSY	1


15. CUSPIDOR SHAFT SECTION

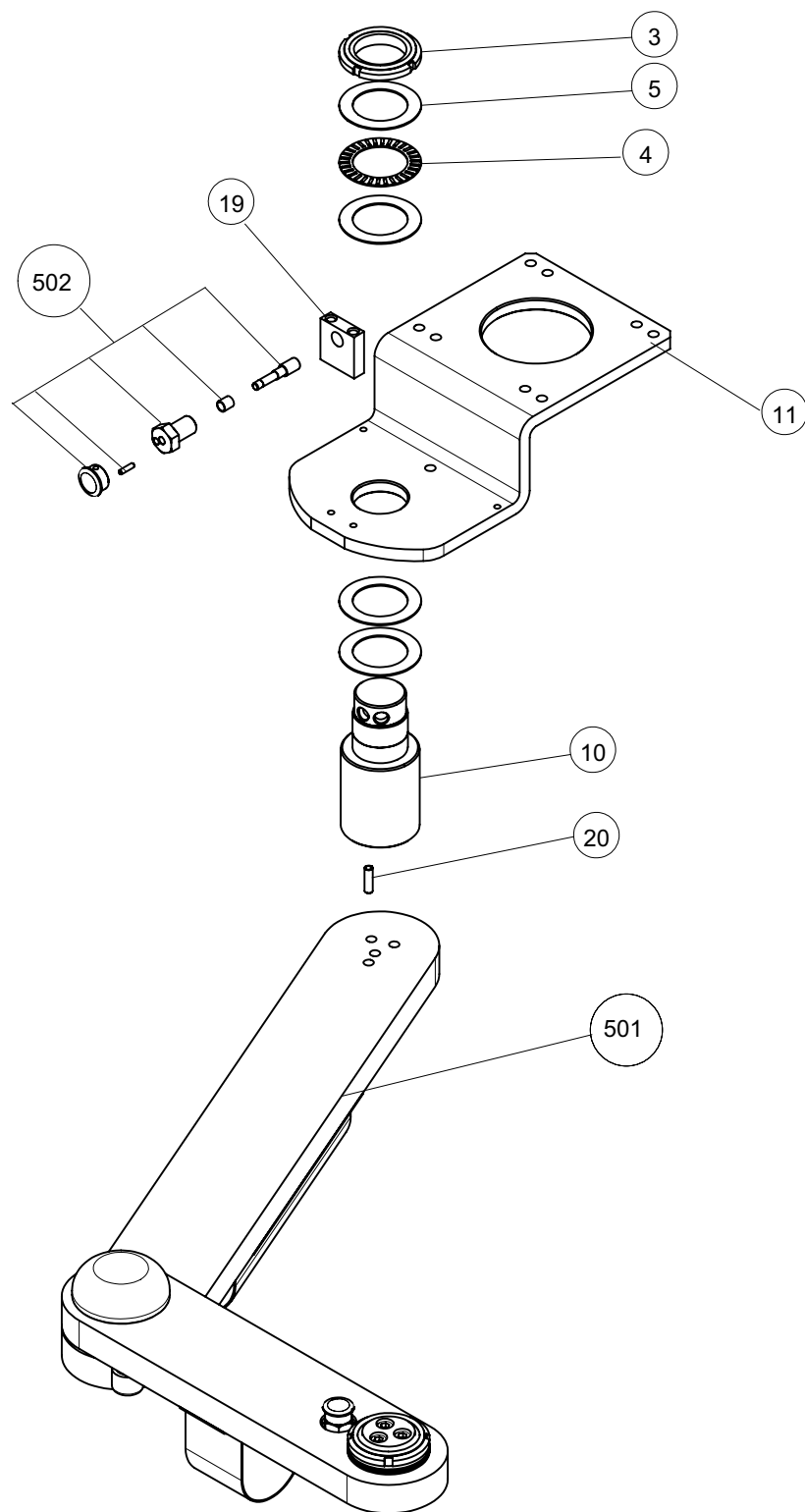

15. CUSPIDOR SHAFT SECTION

ID No.	Parts No.	Description	QTY
3	F17-X-59080	NUT	1
4	F17-X-59090	BEARING	1
5	F17-X-59100	WASHER	4
10	F17-1A09EAB0	CUSPIDOR SHAFT	1
11	F17-1A09EBA0	BRACKET FOR ARM	1
19	F17-1A09FFA0	LOCK SHAFT BRACKET	1
20	F17-ZG081NWG	ROLL PIN M5-20	1
501	F17-1G031CA0	CUSPIDOR ARM ASSY	1
502	F17-1G031DA0	ARM LOCK ASSY	1


16. CUSPIDOR VALVE SECTION

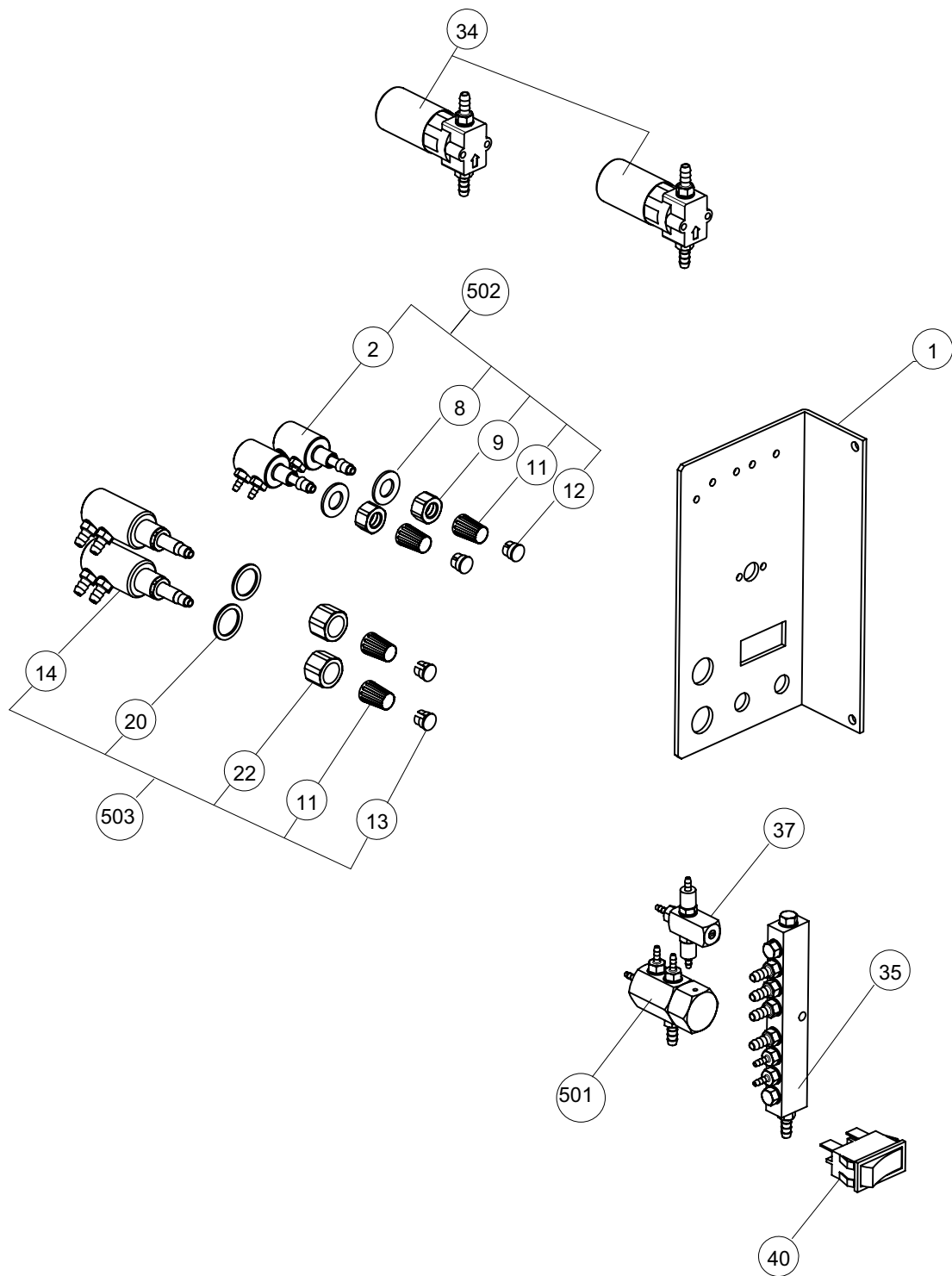

16. CUSPIDOR VALVE SECTION

ID No.	Parts No.	Description	QTY
1	F17-1A09ENA0	BRACKET	1
2	F17-FGTA33A0	NEEDLE VALVE BODY	2
8	F17-ZB561PA	WASHER	2
9	F17-DABS14A0	NUT	2
11	F17-DEMS32AA	KNOB	4
12	F17-X-42200	CAP FOR AIR (YELLOW)	2
13	F17-X-42190	CAP FOR WATER (BLUE)	2
14	F17-FGMA81A0	WATER CONTROL VALVE	2
20	F17-FALC54A0	WASHER	2
22	F17-DARS36A0	NUT	2
34	F17-1G01PAA0	SOLENOID VALVE FOR CUPFILLER /BOWL FLUSH VDW21-6G-2-M5A	1set
35	F17-AAES38A0	AIR MANIFOLD WITHOUT FITTING	1
37	F17-FGMA04A0	FLOW ADJUSTABLE CHECK VALVE	1
40	F17-DEAS03B0	ROCKER SWITCH AJ8102BF	1
501	F17-1G00AJA0	CONTROL VALVE FOR LINE VALVE	1
502	F17-1P02DFA0	MINI NEEDLE VALVE FOR AIR	2
	F17-APDA11A0	MINI NEEDLE VALVE FOR WATER	2
503	F17-1P02FXA0	WATER CONTROL VALVE	2


17. SOLID COLLECTOR SECTION

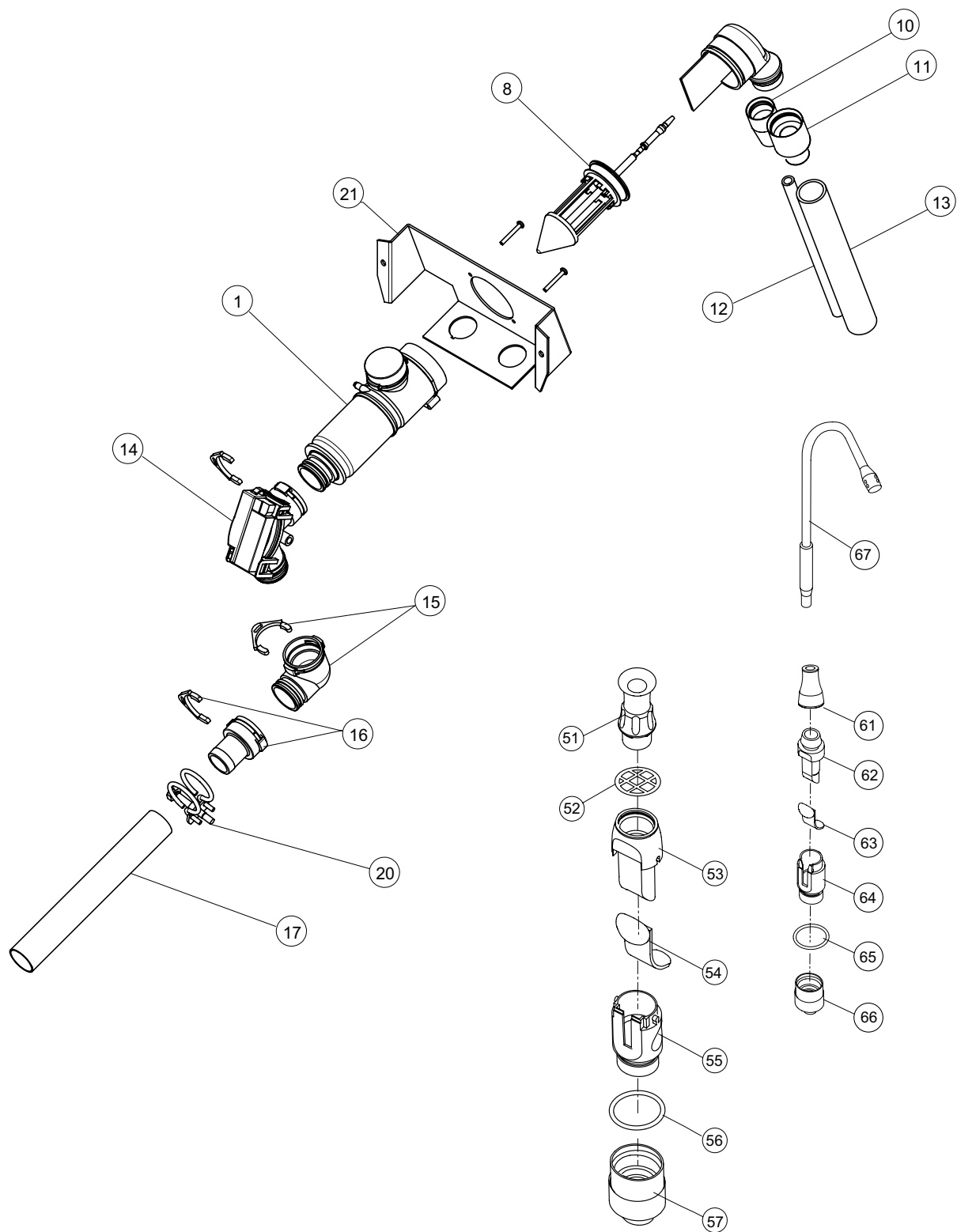

17. SOLID COLLECTOR SECTION

ID No.	Parts No.	Description	QTY
1	F17-1E00KJA0	TWIN SOLID COLLECTOR	1
8	F17-1E00KLB0	SOLID COLLECTOR FILTER	1
10	F17-FABA69A0	SALIVA EJECTOR HOSE CONNECTOR	1
11	F17-FECA17B0	HOSE CONNECTOR	1
12	F17-1P00J7A0	SALIVA EJECTOR HOSE (VOHRAN HOSE ID8-1.5M)	1.5M
13	F17-FEMA20B0	VACUUM HOSE (VOHRAN HOSE ID17.3-1.5M)	1.5M
14	F17-1E00AHA0	SELECTION VALVE	1
15	F17-AFDR29A0	JOINT DURR 0700-700-04E	1
16	F17-AFDR28A0	JOINT DURR 0700-720-20E	2
17	F17-DETR03A0	DRAIN HOSE (ID19)	8
20	F17-X-42270	HOSE CLAMP 18-25-OX	2
21	F17-1A09EKA0	SOLID COLLECTOR BRACKET	1
51	F17-FADA50A0	ROTATION JOINT (ID11)	1
		ROTATION JOINT (ID15.7)	1
52	F17-FAEA41A0	FILTER FOR ROTATION JOINT (ID11)	1
53	F17-FADA48A0	UPPER VACUUM HANDPIECE BODY	1
54	F17-FADA49B0	SLIDE VALVE	1
55	F17-FADA47C0	VACUUM HANDPIECE BODY	1
56	F17-FJDA09A0	O-RING	1
57	F17-FHFA72A0	VACUUM HOSE JOINT	1
61	F17-FADA59A0	TIP JOINT	1
62	F17-FADA56A0	UPPER SALIVA EJECTOR HANDPIECE BODY	1
63	F17-FADA58B0	SLIDE VALVE	1
64	F17-FADA55A0	SALIVA EJECTOR HANDPIECE BODY	1
65	F17-FJDA11A0	O-RING S14	1
66	F17-FHFB23A0	HOSE CONNECTOR WITH CAP (ID8.5)	1
67	F17-FAFA25A0	SALIVA EJECTOR TIP	1


18. ASSISTANT ARM (STANDARD) SECTION

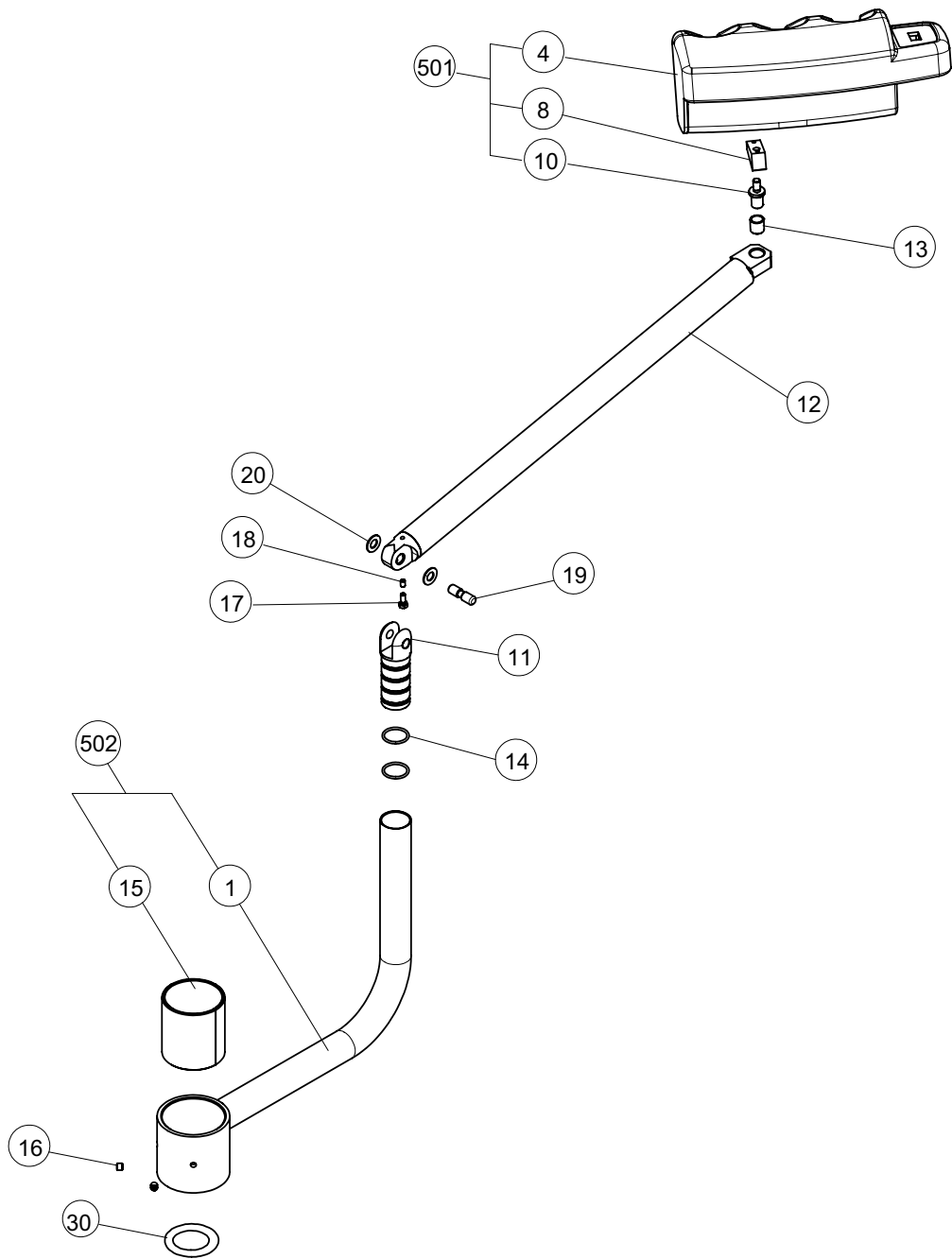

18. ASSISTANT ARM (STANDARD) SECTION

ID No.	Parts No.	Description	QTY
1	F17-1A09EFA0	LOWER ASSISTANT ARM	1
4	F17-1G0104A0	ASSISTANT HOLDER ASSY FOR CENTRAL	1
8	F17-DATT23A0	BRACKET	1
10	F17-DATR45B0	SHAFT	1
11	F17-1A09ECA0	ROTATION SHAFT	1
12	F17-1A09F4A0	UPPER ASSISTANT ARM	1
13	F17-AESA05C0	DU BUSH K5B1012	1
14	F17-AEER02A0	O-RING 1.9-19.8-23.6	2
15	F17-FJAA09C0	DU BUSH K5B5060	1
16	F17-ZA7740L	SET SCREW M6-5	2
17	F17-ZA80ALPU	BOLT M4-8	1
18	F17-ZA774LN	SET SCREW M4-6	1
19	F17-1A09E7A0	PIN FOR UPPER ASSISTANT ARM	1
20	F17-AAFX89A0	WASHER 8-6-1.0	1
30	F17-1A09EDH0	WASHER	1
501	F17-AX0A00A0	ASSISTANT HOLDER ASSY	1
502	F17-1G0373A0	ARM ASSY	1


19. ASSISTANT ARM (BALANCE ARM TYPE) SECTION

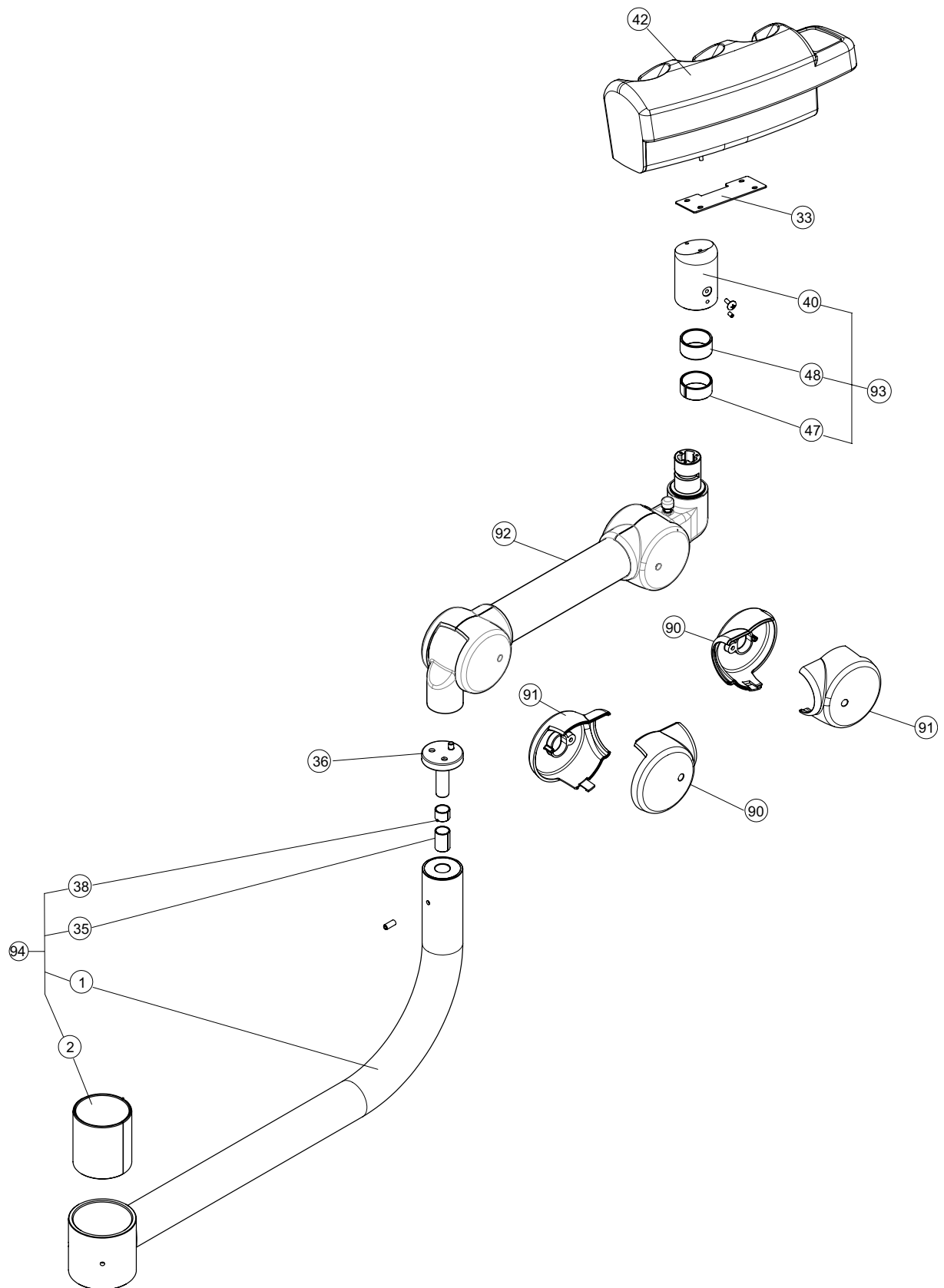

19. ASSISTANT ARM (BALANCE ARM TYPE) SECTION

ID No.	Parts No.	Description	QTY
1	F17-1A0AG9A0	FIRST ARM FOR RATCHET TYPE	1
2	F17-FJAA09B0A	DU BUSH (DAIDOU K5B5060)	1
33	F17-FACC02B0	HOLDER REINFORCED PLATE	1
35	F17-AESA86C0	DU BUSH	1
36	F17-1A0AD2A0	WASTE RECEPTACLE HOLDER	1
38	F17-AEES48B0	DU BUSH (DAIDOU K5B1210)	1
40	F17-1A0AGAA0	HOLDER ROTATION BRACKET	1
42	F17-AXOA00A0	ASSISTANT HOLDER ASSEMBLY (CENTRAL TYPE)	1
47	F17-1A01AKA0	BRAKE SHAFT (26)	1
48	F17-DEMR42C0	DU BUSH (DAIDOU K5B2615)	1
90	F17-AAFF02A0	SUPPORT CAP A	2
91	F17-AAFF03A0	SUPPORT CAP B	2
92	F17-1G047BA0	BALANCE ARM ASSY	1
93	F17-1P03ASA0	ROTATION BRACKET ASSY	1
94	F17-1G037RA0	FIRST ARM ASSY FOR RATCHET TYPE	1


20. CUSPIDOR BOWL SECTION

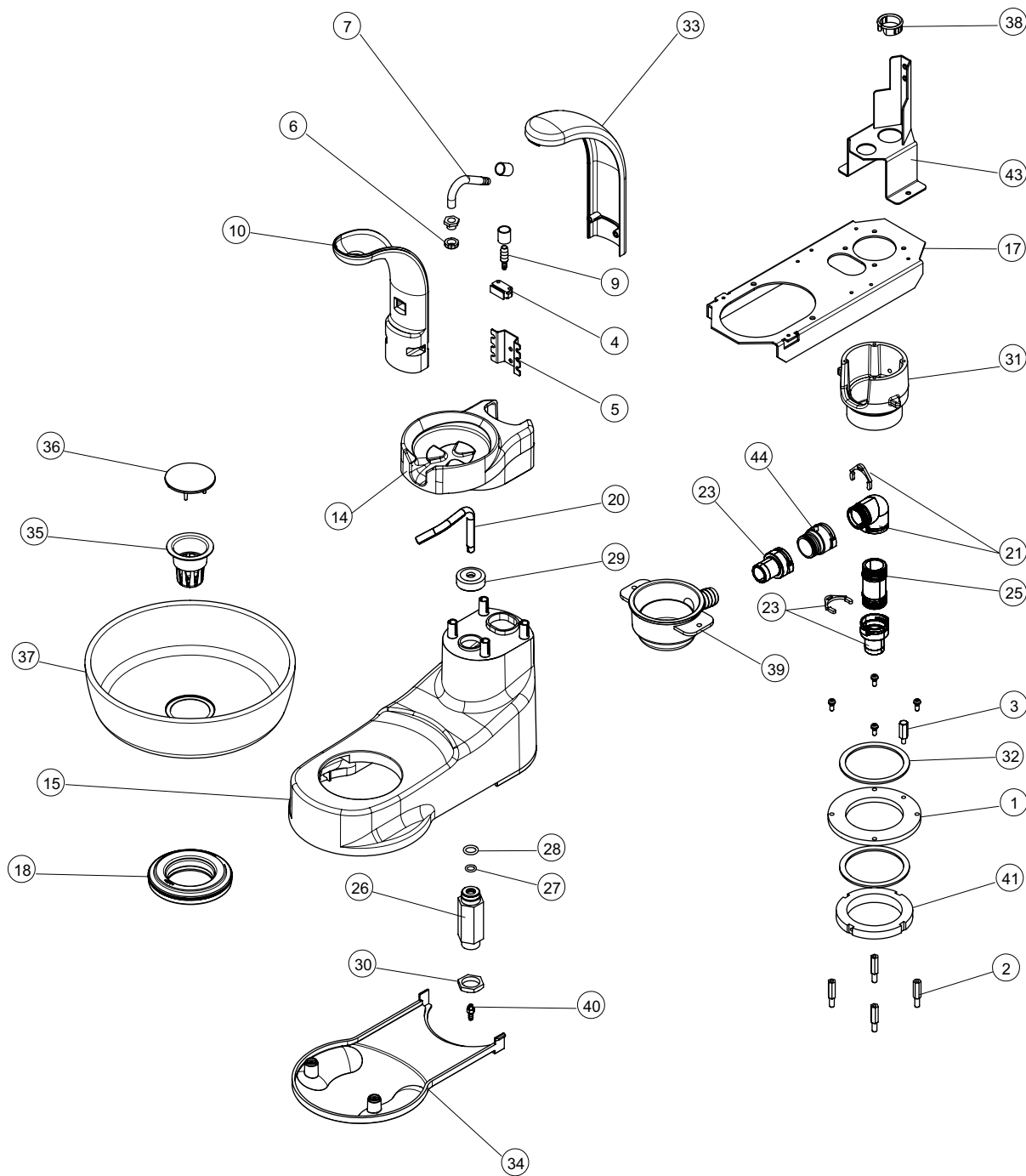

20. CUSPIDOR BOWL SECTION

ID No.	Parts No.	Description	QTY
1	F17-1A09EHA0	SPITTOON ROTATION BRACKET	1
2	F17-1A09EJA0	SPITTOON SPACER	4
3	F17-FACA40A0	ROTATION STOPPER BOLT	1
4	F17-AEER81A0	CUPFILLER SENSOR(OPTION)	1
5	F17-FACA34A0	CUPFILLER NOZZLE FIXING PLATE	1
6	F17-FACA41A0	CUPFILLER MOUTH	1
7	F17-FACA42B0	CUPFILLER NOZZLE	1
9	F17-FAQA11B0	SUPPLY JOINT	1
10	F17-FGCA02C0	CUPFILLER FRONT COVER SENSOR TYPE	1
14	F17-FACA06B0	CUP BASE	1
15	F17-FACA07B0	CUSPIDOR UPPER COVER	1
17	F17-1A0ABTA0	CUSPIDOR BOWL BRACKET	1
18	F17-AAFS22A0	BOWL PACKING	1
20	F17-FACA31A0	BOWL FLUSH NOZZLE	1
21	F17-AFDR29A0	DRAIN JOINT 0700-720-04E	2
23	F17-AFDR28A0	DRAIN JOINT 0700-720-20E	2
25	F17-FACA25A0	STRAIGHT JOINT	1
26	F17-FACA29A0	BOWL FLUSH BASE HOLDER	1
27	F17-1E01PNA0	O-RING P-8	1
28	F17-AFDR36A0	O-RING P-11	1
29	F17-FACA30A0	NOZZLE WASHER	1
30	F17-DAOR93B0	BOWL FLUSH BASE HOLDER NUT	1
31	F17-FACA36B0	ROTATION RING	1
32	F17-FACA39A0	ROTATION RING WASHER	2
33	F17-FACA04B0	CUPFILLER REAR COVER	1
34	F17-FACA08B0	CUSPIDOR LOWER COVER	1
35	F17-FALE80A0	DRAIN STRAINER	1
36	F17-FALG93A0	DRAIN CAP	1
37	F17-FHGA45A0	CERAMICS CUSPIDOR BOWL ASSY	1
	F17-FACA37B0	GLASS CUSPIDOR BOWL (BLUE)	1
	F17-FACA58B0	GLASS CUSPIDOR BOWL (CLEAR)	1
	F17-FACA65B0	GLASS CUSPIDOR BOWL (GREEN)	1
	F17-FACA66B0	GLASS CUSPIDOR BOWL (PINK)	1
38	F17-XX-387240352	BUSH OCB1000	1
39	F17-FACA02A0	DRAIN CUP	1
40	F17-AAES69B0	BARB FITTING BF1/4	1
41	F17-DATS82A0	ROTATION RING NUT AN12	1
43	F17-1A07C0A0	CUPFILLER BRACKET	1
44	F17-1A0367A0	DRAIN JOINT 0700-700-15E	1


21. JUNCTION BOX SECTION

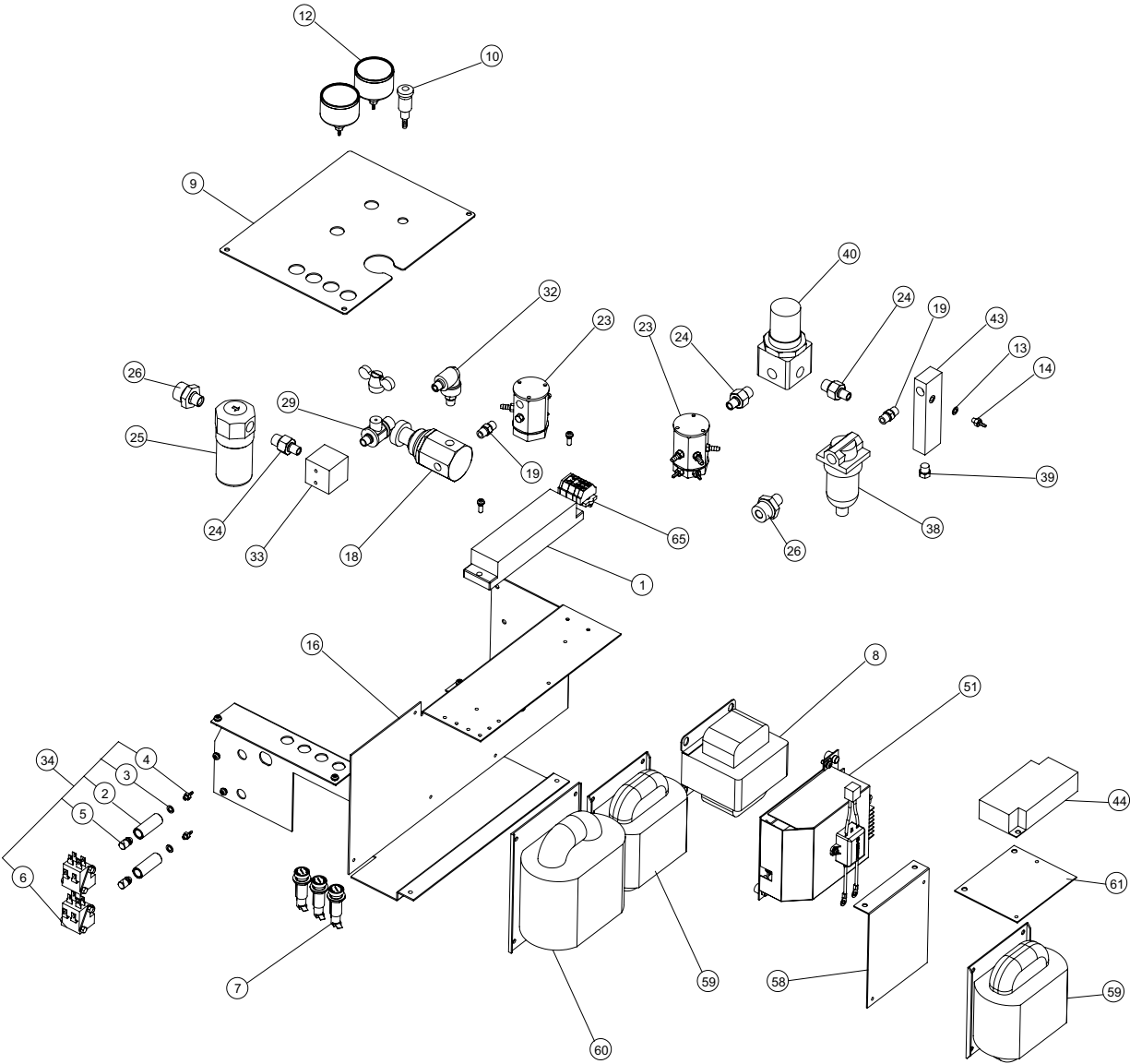

21. JUNCTION BOX SECTION

ID No.	Parts No.	Description	QTY
1	F17-DNBR18B0	TERMINAL BLOCK BTB 15LC-12	1
2	F17-FALC30A0	AIR SWITCH BODY	2
3	F17-AAES67A0	NYLON WASHER M5	2
4	F17-AAES68A0	BARB FITTING 1/8	2
5	F17-FALC32A0	AIR SWITCH SPOOL	2
6	F17-X-5534	MICRO SWITCH (2VAQ-4)	2
7	F17-AEFR93A0	FUSE HOLDER	3
	F17-DERR49A0	FUSE 5A/250V	1
	F17-CESN21B0	FUSE 6.3A(CE)	2
8	F17-1G03PBA0	720 DENTAL LIGHT TRANSFORMER 115V	1
	F17-1G03AJA0	720 DENTAL LIGHT TRANSFORMER 220V	1
	F17-1G032TA0	720 DENTAL LIGHT TRANSFORMER 230V	1
	F17-1G03AKA0	720 DENTAL LIGHT TRANSFORMER 240V	1
	F17-1G03PCA0	820 DENTAL LIGHT TRANSFORMER 115V	1
	F17-1G03ALA0	820 DENTAL LIGHT TRANSFORMER 220V	1
	F17-1G032UA0	820 DENTAL LIGHT TRANSFORMER 230V	1
	F17-1G03AMA0	820 DENTAL LIGHT TRANSFORMER 240V	1
9	F17-1A09F2A0	JUNCTION PANEL	1
10	F17-FHNA11A0	AIR DRAIN VALVE	1
12	F17-DESR14A0	PRESSURE GAUGE 18-013-989-Z2	2
	F17-DASR14B0	PRESSURE GAUGE NUT	2
13	F17-AAES67A0	NYLON WASHER M5	1
14	F17-AAES68A0	BARB FITTING 1/8	6
16	F17-1A09E6A0	JUNCTION BRACKET	1
18	F17-AEFU69B0	WATER REGULATOR WR 1-6-N	1
19	F17-AAES10B0	JOINT 1/8	3
23	F17-DGRR02D0	MAIN AIR MASTER VALVE	1
24	F17-AAES33B0	JOINT 1/4-1/8	3
25	F17-AEFU71A0	WATER FILTER A1019-2C-MS18	1
26	F17-FALC19A0	NIPPLE 1/4-1/2	2
29	F17-1E0059A0	BALL VALVE BM-9011R	1
32	F17-AAES19A0	STREET ELBOW 1/8-1/8	1
33	F17-FACB58A0	BALL VALVE BLOCK	1
34	F17-1P02FVA0	AIR SWITCH ASSY BF 1/8 FOR MAIN SWITCH	1
34	F17-1P02FUA0	AIR SWITCH ASSY BF 1/4 FOR VACUUM OPERATION	1
38	F17-AEFU70A0	AIR FILTER A1019-2CMS23	1
39	F17-AAES34B0	PLUG PT 1/8	1
40	F17-AEFU68A0	AIR REGULATOR B2019-2C	1
43	F17-FACB60B0	AIR MANIFOLD	1
44	F17-1A09NTA0	NOISE FILTER ASSY	1
	F17-1E02TAA0	NOISE FILTER MB1206	1
51	F17-1G037GB0	LED LIGHT(920) PCB COVER	1
58	F17-1A09FGA0	TRANSFORMER BRACKET(6-WAY SYRINGE)	1
59	F17-1G03PGA0	TRANSFORMER FOR 6-WAY SYRINGE (NK-020)115V	1
	F17-1G03PHA0	TRANSFORMER FOR 6-WAY SYRINGE (FK-018)220V	1
	F17-1G032VA0	TRANSFORMER FOR 6-WAY SYRINGE (FK-018)230V	1
	F17-1G03PJA0	TRANSFORMER FOR 6-WAY SYRINGE (FK-018)240V	1
60	F17-1G00V9B0	UNIT TRANSFORMER 115V	1
	F17-1G00XBA0	UNIT TRANSFORMER 220V	1
	F17-1G00QJA0	UNIT TRANSFORMER 230V	1
	F17-1G00XCA0	UNIT TRANSFORMER 240V	1
61	F17-1A09Q9A0	NOISE FILTER BRACKET	1
65	F17-1G00UEA0	TERMINAL CLAMP ASSY	1


22. LIGHT POLE SECTION

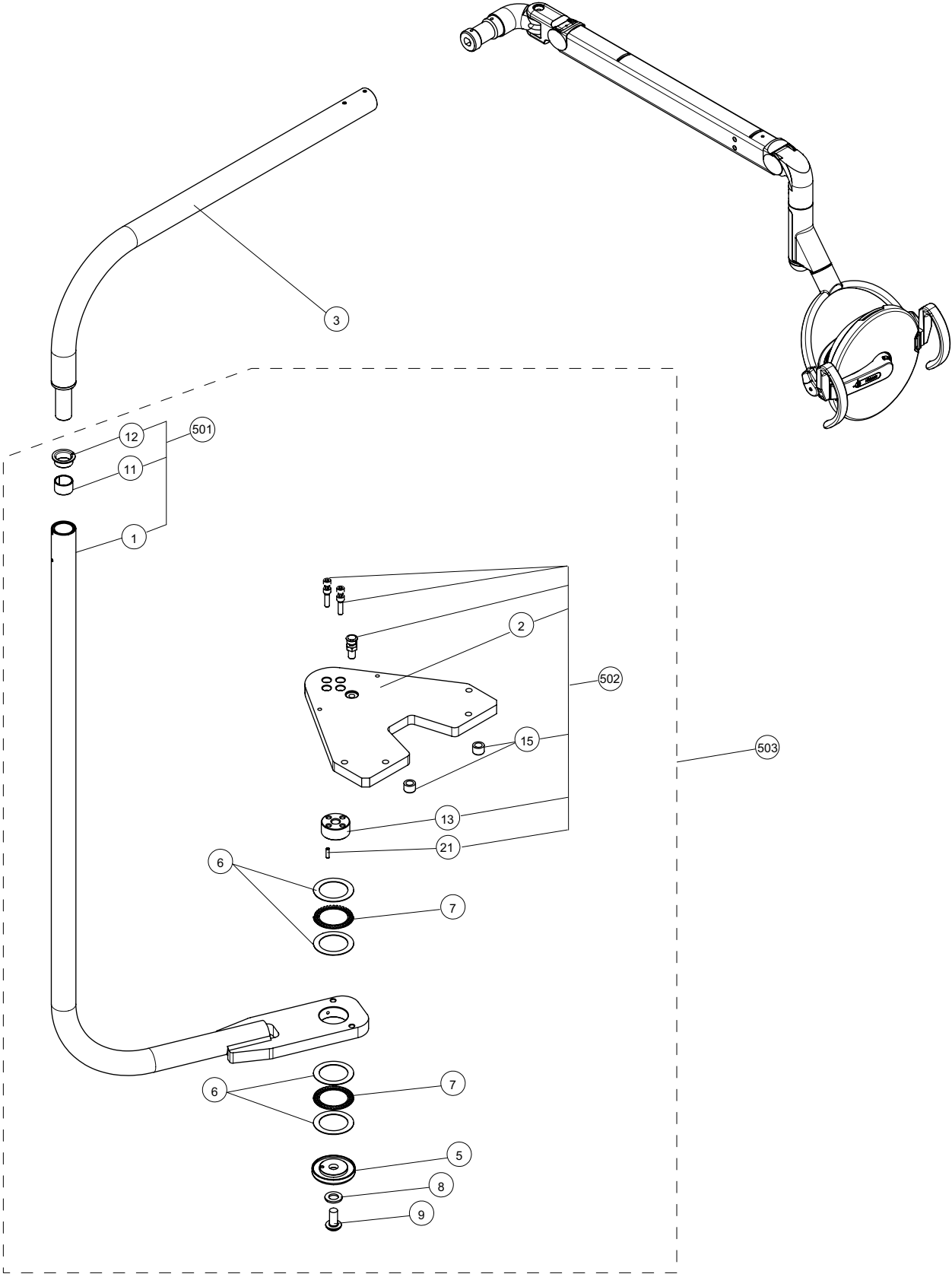

22. LIGHT POLE SECTION

ID No.	Parts No.	Description	QTY
1	F17-1A09ETA0	LIGHT ARM	1
	F17-1A0ADXA0	LIGHT ARM FOR MONITOR BRACKET	1
2	F17-1G0327A0	MOUNTING BRACKET	1
3	F17-1A0ADWA0	L-TYPE LIGHT POLE 720/820/920 DENTAL LIGHT	1
5	F17-AAFT92A0	SWING ARM ROTATION WASHER	1
6	F17-AFFS40A0	THRUST WASHER AS1110	4
7	F17-AFFS41A0	THRUST BEARING AXK1110	2
8	F17-1E00MGA0	FLAT WASHER M16 X 32 2.6	1
9	F17-ZSA11TZU	BUTTON CS SCREW (M16 X 30 SUS)	1
11	F17-AEFU14A0	BUSH K5B2820	1
12	F17-AEFU13A0	BUSH K5F2812-40	1
13	F17-AAFT93A0	UPPER ARM SHAFT	1
15	F17-DAMS68AA	STOPPER RUBBER	2
21	F17-ZG081N#G	ROLL PIN M5-15	1
501	F17-1P02T5A0	LIGHT SWING ARM ASSY WITHOUT MONITOR BRACKET	1
	F17-1P02T6A0	LIGHT SWING ARM ASSY WITH MONITOR BRACKET	1
502	F17-1G0327A0	MOUNTING BRACKET ASSY	1
503	F17-1G03ANA0	LIGHT POLE ASSY WITHOUT MONITOR BRACKET	1
	F17-1G036RA0	LIGHT POLE ASSY WITH MONITOR BRACKET	1

23. VAC PACK SECTION

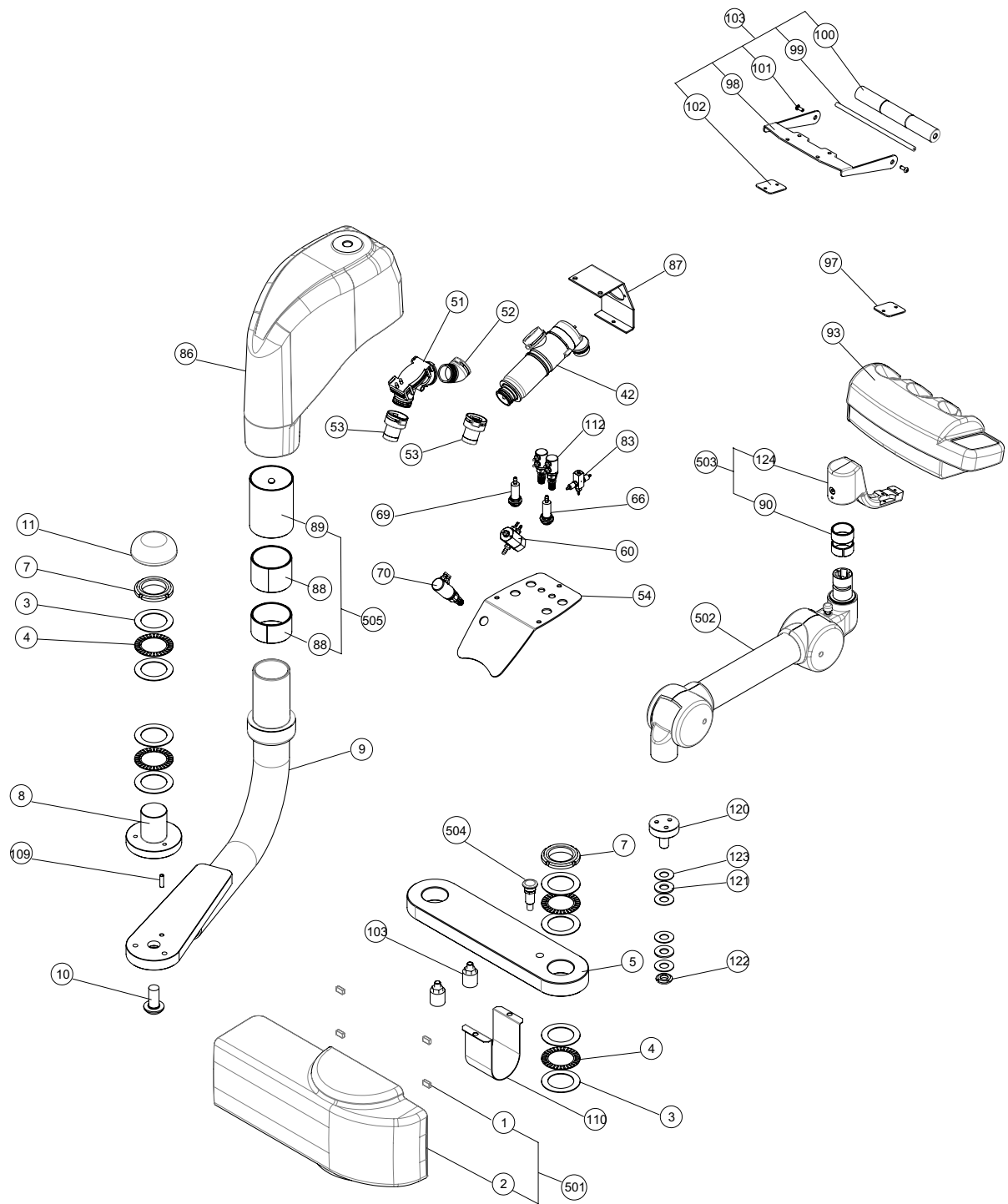

23. VAC PACK SECTION

ID No.	Parts No.	Description	QTY
1	F17-1A09QAA0	MAGNET	4
2	F17-1A0A1BA0	BACK PAC BRACKET COVER	1
3	F17-X-59100	WASHER (FTRA3552)	8
4	F17-X-59090	BEARING (FTRA3552)	4
5	F17-1A09E9A0	CUSPIDOR FIRST ARM	1
7	F17-X-59080	NUT (FU07M35XP1.5)	2
8	F17-1A0CS5A0	BACK PAC ARM ROTATION SHAFT	1
9	F17-1A0CS6A0	BACK PAC SECOND ARM	1
10	F17-ZSA11TZU	BUTTON CS (M16-30 SUS)	1
11	F17-AAFT58A0	RUBBER CAP	1
42	F17-1E00KJA0	SOLID COLECTOR (DURR TYPE)	1
51	F17-1E00AHA0	CUT OFF VALVE	1
52	F17-1E03W9A0	JOINT (0700-700-05E)	1
53	F17-AFDR28A0	JOINT (0700-720-20E)	2
54	F17-1A0CS7A0	BRACKET FOR VALVE	1
60	F17-1G00AJA0	VACUUM OPERATING VALVE	1
66	F17-AEFR04A0	SURVICE CUPLER FOR AIR (MC-04SHS)	1
69	F17-1E01L6A0	SURVICE CUPLER FOR WATER (MCL-04BH-1B)	1
70	F17-FGMA81A0	ADJUSTMENT VALVE FOR WATER	1
83	F17-FGMA04A0	ADJUSTMENT VALVE WITH CHECK VALVE	1
86	F17-1A0CS8A0	BACK PAC BODY	1
87	F17-1A0CS9A0	BRACKET FOR SOLID COLECTOR	1
88	F17-AFDR33C0	DU BUSH (DAIDOU K5B5030)	2
89	F17-1A0CSDA0	ROTATION BEARING FOR BACK PAC BODY	1
90	F17-DERR57C0	DU BUSH (K5B2620)	1
93	F17-1G0475A0	ASSISTANT HOLDER ASSEMBLY (CENTRAL TYPE)	1
97	F17-DATT24B0	BRACKET FOR ASSISTANT HOLDER	2
98	F17-1A0CSBA0	ROLLER BRACKET	1
99	F17-1A0CSCA0	ROLLER BAR	1
100	F17-1A08A7A0	VACUUM ROLLER	1
101	F17-ZSA11LR	SET SCREW M4-10	1
102	F17-DATT24B0	BRACKET FOR ASSISTANT HOLDER	1
103	F17-1P0371A0	ROLLER ASSY	1
104	F17-DAMS68AA	STOPPER RUBBER	2
109	F17-ZG081NWG	ROLL PIN 5.0-20 BLACK	1
110	F17-1A09Q7A0	FIRST HOSE COVER FOR	1
112	F17-FGTA33A0	NEEDLE VALVE (SP-M2)	2
120	F17-1A0DG2A0	ARM ROTATION SHAFT	1
121	F17-1E00E5A0	BEARING (□12)	2
122	F17-1E00E6A0	U-NUT (M12 P=1)	1
123	F17-AERN11A0	WASHER (AS1101)	4
124	F17-1A0368A0	HOLDER BRACKET (RALL9003)	1
501	F17-1G047AA0	SWING ARM BRACKET COVER	1
502	F17-1G047BA0	BALANCE ARM ASSY	1
503	F17-1P03ARA0	HOLDER BRACKET ASSY RAL9003	1
504	F17-1G032DA0	LOCK PIN	1
505	F17-1G037VA0	VAC PACK ROTARION BRACKET	1



TAKARA BELMONT CO., LTD.

1-1, Higashi-shinsaibashi 2-Chome, Chuo-ku, Osaka, Japan

TEL. : (06) 213-5945

TELEFAX : (06) 212-3680

Printed in Japan F.EDITION-20140310